The 91443B Flexport® interface integrates data provided by the FLOW-i Anesthesia Delivery System (ADS) into Spacelabs Healthcare XPRESSON™, qube™, or Ultraview SL™ monitors.

The 91443B Flexport interface provides FLOW-i data for use in tabular and graphical trending on a monitor. The Flexport interface provides Remote View capability of the trends from central and remote monitors. The data provided from the Flexport interface is also made available to the Intesys® Clinical Suite (ICS) for display in Clinical Access trends and to interface to clinical information systems. All compatible XPRESSON, qube, and Ultraview SL monitors support multiple Flexport interfaces.

Features

Parameters	Each Flexport interface counts as one parameter when computing parameter capacity for monitors.
------------	---

Product Specifications

Physical Dimensions	
Weight	0.11 kg (4 ounces)
Height	2.5 cm (1 inch)
Depth	4.6 cm (1.8 inches)
Width	13.5 cm (5.3 inches)

Status message

ANES

- Notes:
- No parameters are shown on the display for the 91443B Flexport FLOW-i.
 - For a list of parameters available with the Maquet FLOW-i Anesthesia Delivery System (ADS), refer to the Global Data System (GDS) System Administration Guide (P/N 070-1973-xx).

Environmental Requirements

Storage	
Temperature	-40° to 75° C (5° to 167° F)
Humidity	95% (non-condensing)
Altitude	0 to 12,192 m (0 to 40,000 feet)
Operating	
Temperature	10° to 40° C (50° to 104° F)
Humidity	95% (non-condensing)
Altitude	0 to 2,000 m

Monitor Compability

The 91443B Flexport is supported by the following Spacelabs Healthcare monitors:

XPRESSON™, qube™	XPRESSON (91393), qube (91390)
Ultraview SL™	SL2200 (91367), SL2400 (91369), SL2600 (91370), SL2700 (91387-27), SL2800 (91387-28)
Monitors not supported	Enhanced (90385), Ultraview 1700, Enhanced (90387), Ultraview 1500 (90363), Ultraview 1600 (90364), Ultraview 1050 (90369), Ultraview 1030 (90367), UCW® Classic (90385), Ultraview 1700 Classic (90387), Ultracare SLP100, PC2 (90305), PC1 (90303), PCX (90308), PCSout® (90309), mCare (91220), Maternal Obstetrical Monitor (MOM™) (94000)

Note:

Spacelabs Healthcare confirms that the Flexport software versions shown in the table that follows are verified for use with the software or protocol versions listed for the source devices.

Company, Product, or Source Device	Software or Protocol Version	Spacelabs Healthcare Flexport Software Version	Flexport Model Number
Maquet FLOW-i Anesthesia Delivery System C20, C30, C40	FCI protocol version 0004	1.00.00-009	91443B

Transition Connector

91443B Maquet FLOW-i Flexport	P/N 131-0686-02
-------------------------------	-----------------

Accessories

Connectors	- Female 6-pin modular jack - Male 9-pin SDLC connector
Cable length	- SDLC — 30.48 cm (12 inches) - RS-232 — 243.8 cm (96 inches), P/N 012-0182-02
Maximum SDLC cable length	The total length of all SDLC cables from the power source (monitor or module housing) to the Flexport device interface should not exceed 1.8 m (6 feet). When linking other Spacelabs Healthcare devices, the Flexport device interface must be the closest device to the monitor or module housing.

Documentation

CD-ROM part number	<i>Flexport Maquet FLOW-i Operations and Service CD-ROM (P/N 084-1484-xx)</i>
Supplies and Accessories	Spacelabs Healthcare Supplies and Accessories Catalog (sa.spacelabshealthcare.com)

Regulatory Approvals

CE marked in accordance with the Medical Device Directive 93/42/EEC.

Please refer to <https://www.spacelabshealthcare.com/about-us/patents-trademarks> for a full listing of Spacelabs Healthcare trademarks. Other brands and product names used herein are trademarks of their

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