

SPACELABS
HEALTHCARE




5489-087345

John Doe | 64 years | Male

60  /min

CardioPulse Go

Operating manual

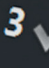1  2  3 

Rhythm

Manu

LP 150Hz, AC 50Hz

REF: 2.511461 REV B
CAT: 070-3062-00 REV B

R_xonly

SPACELABS
HEALTHCARE

All rights reserved. Contents of this publication may not be reproduced in any form without the written permission of Spacelabs Healthcare. Products of Spacelabs Healthcare are covered by U.S. and foreign patents and/or pending patents.

Spacelabs Healthcare considers itself responsible for the effects on safety, reliability and performance of the equipment only if:

- assembly operations, re-adjustments, modifications or repairs are carried out by persons authorized by Spacelabs Healthcare, and
- the electrical installation of the relevant room complies with the requirements of the standard in force, and
- the equipment is used in accordance with the operations manual.

In the event of a serious incident, notify Spacelabs and the competent authority.

Spacelabs Healthcare will make available, on request, component part lists, descriptions, calibration instructions or other information which will assist appropriately qualified technical personnel to repair those parts of the equipment which are classified by Spacelabs Healthcare as field repairable.

Spacelabs Healthcare is committed to providing comprehensive customer support beginning with your initial inquiry through purchase, training, and service for the life of your Spacelabs Healthcare equipment.



Distributed by

Spacelabs Healthcare
35301 SE Center Street
Snoqualmie, WA 98065
Telephone: (1) 800-287-7108
Telephone: (1) 425-396-3300



Manufactured by:

SCHILLER AG
Altgasse 68
CH-6341 Baar, Switzerland



The **CardioPulse Go** (Type CARDIOVIT FT-1) bears the CE-0123 mark (Notified Body TÜV-SÜD Produkte Service GmbH, Ridlerstr. 65, 80339 Munich, Germany), indicating its compliance with the essential requirements of the Annex I of the Medical Device Directive 93/42/EE regarding safety, functionality and labelling. The requirements apply to patients, users and third persons who come into contact with this device within the scope of its intended use.

Rx_{only}

Caution: Federal law restricts this device to sale by or on the order of a physician.



Contents

1	Safety Notes	7
1.1	Intended purpose	7
1.2	Contraindications	8
1.3	Responsibility of the User	8
1.4	Organizational Measures	9
1.5	Safety-Conscious Operation	9
1.6	Safety facilities	10
1.7	Operation with other Devices	10
1.8	Network safety	11
1.9	Maintenance	11
1.10	Serious incidents	11
1.11	Terms of Warranty	11
1.11.1	Additional statements	12
1.12	Safety Symbols and Pictograms	13
1.12.1	Symbols used in this document	13
1.12.2	Symbols used on the device	14
2	Introduction	15
2.1	Main components of the CardioPulse Go	15
2.1.1	Standard	16
2.1.2	Options	16
2.2	Connections	16
2.3	Display	17
2.4	Multi-touch screen operation	18
2.5	Connection panel	20
2.5.1	Back panel	20
2.5.2	Right hand side panel	20
3	Operation	21
3.1	Start-up and Preparation	21
3.1.1	Location	21
3.1.2	Connection of external cable assemblies and ancillary equipment	21
3.1.3	Potential equalisation	21
3.2	Switching on / off	22
3.2.1	Login and Logout / Emergency ECG	22
3.3	Power supply	23
3.3.1	AC power and battery indicators	23
3.3.2	Isolating the device from the AC power supply	23
3.4	System and ECG settings	24
3.5	Changing the printing paper	25

3.6	Patient / recording data.....	26
3.6.1	Patient data query (PDQ)	29
3.6.2	PDQ in the worklist	29
3.6.3	Barcode reader	30
4	Placing the electrodes	31
4.1	Basics	31
4.2	Electrode Identification and Color Code	32
4.3	Standard 10-lead resting ECG	33
4.3.1	Electrode placement for standard leads	33
4.4	Balanced.....	34
4.5	Pediatric.....	35
4.6	Right precordials (V3r-V6r).....	36
4.7	Standard (V4r).....	37
4.8	Left posterior V7-V9.....	38
4.9	Nehb leads.....	39
4.10	Skin/Electrode Resistance.....	40
4.10.1	Electrode and patient cable check.....	40
4.11	Lead sequence/lead view.....	41
4.11.1	Setting Standard or Cabrera lead sequence	41
4.11.2	Select the lead view (Standard or other settings).....	41
5	Resting ECG	42
5.1	Resting ECG - procedural flow diagram.....	43
5.1.1	Printing, saving and transferring automatically.....	44
5.2	Automatic resting ECG recording.....	45
5.2.1	Automatic printout.....	46
5.3	Manual rhythm printout.....	47
5.3.1	Starting manual printout	47
5.4	Rhythm recording.....	48
5.4.1	Automatic printout.....	49
5.5	Changing the ECG preview layout.....	50
5.5.1	Display.....	50
5.5.2	Myogram filter (Low pass filter)	51
5.5.3	Other filters	51
5.6	ECG settings	52
5.6.1	Leads & cable.....	52
5.6.2	Filters & formulas.....	53
5.6.3	Views & layouts	54
5.6.4	Additional leads	57
5.6.5	Interpretation	57
5.6.6	Resting rhythm	57
5.6.7	Reports	58
6	Memory	62
6.1	Editing the memory	62
6.1.1	Opening the print preview from the memory and printing a recording.....	63
6.1.2	Transmitting and deleting stored recordings	64

7	Worklist (Option)	65
7.1	General information	65
7.1.1	Worklist settings	65
7.2	Receiving a worklist	66
7.2.1	Taking a Worklist Recording	67
7.2.2	Performing a recording from work order details	68
7.2.3	Sending worklist recordings to the HIS	69
8	General and system settings	70
8.1	System settings	70
8.1.1	Saving and restoring settings	71
8.2	ECG	72
8.2.1	Leads & cable	72
8.2.2	Filters & formulas	72
8.2.3	Views & layouts	73
8.2.4	Additional leads	74
8.2.5	Interpretation	74
8.2.6	Resting rhythm	74
8.2.7	Reports	75
8.3	Connectivity	78
8.4	Regional settings	79
8.5	General	80
8.5.1	Setting user name and password for access control	82
9	Transmission - Overview	83
9.1	Transmission Options	83
9.1.1	Automatic transmission	84
9.1.2	Manual transmission	84
9.1.3	PDF export	85
9.1.4	Schiller Link	86
9.1.5	Receiving data from server	86
9.1.6	Failed data transmission	87
10	Maintenance	88
10.1	Visual Inspection	88
10.2	Cleaning the casing and cables	89
10.2.1	Cleaning the cable assembly	90
10.2.2	Approved cleaning solutions	90
10.2.3	Non-admissible detergents	90
10.3	Disinfection	91
10.3.1	Approved disinfectants	91
10.3.2	Non-admissible disinfectants	91
10.4	Cleaning the thermal print head	91
10.5	Battery	92
10.5.1	Charging the battery	92
10.5.2	Battery disposal	92
10.6	Inspection report	93
10.6.1	Lifed-item replacement every 3 - 5 years	94

11	Trouble shooting	95
11.1	Possible problems	95
11.2	Preventing electromagnetic interferences	97
11.2.1	Measures to prevent electromagnetic interferences	98
11.3	Accessories and disposables	99
11.3.1	ECG CablePatient cables	99
11.3.2	Electrodes and ECG consumables	99
11.3.3	AC power cable, Power supply and potential equalisation	99
11.3.4	Other accessories	100
11.3.5	Trolley	100
12	Technical data	101
12.1	Device	101
12.2	ECG	103
12.3	Safety standards	104
12.4	WLAN standards	104
12.5	EMC information	105
12.5.1	Electromagnetic emissions	105
12.5.2	Immunity	105
12.5.3	Emissions equipment and systems	106
12.5.4	Immunity to proximity fields from RF wireless communications equipment	107
12.5.5	Recommended separation distances	108
13	Index	111
14	Appendix - Symbols	113

1 Safety Notes



- ▲ Read and follow these safety notes to prevent any injuries or damages.

1.1 Intended purpose



- ▲ The CardioPulse Go is a 12-lead electrocardiograph device intended to be used by or under the direct supervision of a licensed healthcare practitioner in healthcare facilities to acquire ECG signals from body surface electrodes, record, analyse, display and print ECGs for diagnosis in adult and paediatric patients.

Indications

- ▲ The CardioPulse Go is indicated for screening and assessment of cardiovascular diseases including:
 - Resting myocardial ischaemia
 - Myocardial infarction (acute and former)
 - Conduction system abnormalities including atrio-ventricular blocks, bundle branch block and pre-excitation syndromes
 - Long QT syndromes
 - Atrial abnormalities
 - Ventricular hypertrophy and strain
 - Pericarditis
 - Secondary repolarisation abnormalities such as electrolyte disturbances
 - Drug-induced abnormalities

Intended users

- ▲ The CardioPulse Go is intended to be used by trained operators under supervision of a licensed healthcare practitioner.

Patient target group

- ▲ The CardioPulse Go is intended to be used for adult and paediatric patients.
- ▲ There are no restrictions regarding height, weight, gender or ethnicity of the patients.
- ▲ Paediatric patients are defined as follows:
 - Neonates: from birth through the first 28 days of life
 - Infants: 29 days of age to less than two years of age
 - Children: Two years of age to less than 12 years of age
 - Adolescents: 12 years of age through 21 years of age (up to, but not including, the twenty-second birthday)
- Subgroups:
 - Transitional Adolescent B: 18 through 21 years of age BUT no special considerations compared to adults



Affected body regions

- ▲ The CardioPulse Go is not in direct contact with the patient.

Context of use

- ▲ The CardioPulse Go is intended for indoor use in healthcare facilities.
- ▲ The CardioPulse Go needs to be operated according to the technical data (see [Technical data](#)).
- ▲ The CardioPulse Go is portable. The CardioPulse Go is transportable by means of a trolley (optional).

1.2 Contraindications



System

- ▲ The CardioPulse Go is not indicated:
 - for sterile use
 - for use in areas where there is any danger of explosion or in the presence of flammable gases such as anaesthetic agents
 - for direct cardiac application
 - for use in an MRI suite
 - for outdoor use
 - for use as a vital signs physiological monitor
 - for use with high frequency surgical units



Patient

- ▲ No known contraindications for resting ECG.

1.3 Responsibility of the User



- ▲ The CardioPulse Go must only be used by qualified physicians or trained medical personnel.
- ▲ The numerical and graphical results as well as any interpretation suggested by the device must be examined with respect to the patient's overall clinical condition and the quality of the recorded data.
- ▲ The responsibilities of the personnel for the operation and maintenance of the device must be specified.
- ▲ Ensure that the personnel have read and understood this user guide, in particular this section **Safety Notes**.
- ▲ Damaged or missing components must be replaced immediately.
- ▲ The safety, reliability and performance of the device can only be guaranteed when the maintenance intervals as stated in the section [Maintenance](#) are observed.

1.4 Organizational Measures




- ▲ Before using the device, ensure that a medical product representative has explained its functions as well as the safety requirements.
- ▲ Keep this user guide in an accessible place for reference purposes. Make sure that it is always complete and legible.
- ▲ Observe the operating and maintenance instructions.
- ▲ In addition to this user guide, legal and other binding regulations for the prevention of accidents and for environment protection must be observed.

Packaging

- ▲ Do not use the device and disposables if the packaging is damaged or has been unintentionally opened before.
- ▲ Do not use the device if the packaging is exposed to environmental conditions outside of those specified ([section 12.1 Device, page 101](#)) and contact SCHILLER.

1.5 Safety-Conscious Operation



- ▲ Make sure that the staff have read and understood the operating instructions, in particular this section Safety Notes.
- ▲ Only operate the device in accordance with the specified technical data.
- ▲  The device is CF classified. It is defibrillation protected only when the Spacelabs Healthcare original patient cable is used. However, as a safety precaution, remove the electrodes before defibrillation, if possible.
- ▲ Do not touch the unit casing during defibrillation.
- ▲ To ensure patient safety, none of the electrodes, including the neutral electrode, nor the patient or any person with simultaneous patient contact, must come in contact with conductive parts, even when these are earthed.
- ▲ Immediately report any changes that impair safety (including operating behavior) to the person responsible.
- ▲ Do not place any liquids on the unit. If liquid is spilled on the device, immediately disconnect the device from the mains and wipe it. The device must be serviced before reusing.
- ▲ Only connect the original Spacelabs Healthcare patient cable to the patient socket.
- ▲ If the patient cable should become defective after defibrillation, an electrode becomes displaced, or an electrode resistance is too high, a lead-off indication is displayed in the upper right part of the screen.
- ▲ Only use accessories and disposables recommended or supplied by Spacelabs Healthcare. Use of other than recommended or supplied parts may result in injury, inaccurate information and/or damage to the unit.
- ▲ To prevent pacemaker malfunction, a distance of at least 7.9 inches must be observed between the device and the pacemaker as soon as the Wi-Fi (wireless LAN) module is switched on.


1.6 Safety facilities



- ▲ Operating the device without the correctly rated fuse or with defective cables constitutes a danger to life! Therefore:
 - Do not operate the unit if the earth connection is suspect or if the mains lead, the power supply unit or the device is damaged or suspected of being damaged.
 - Damaged cable connections and connectors must be replaced immediately.
 - The electrical safety devices, such as fuses, must not be altered.
 - Ruptured fuses must only be replaced with the same type and rating as the original.

1.7 Operation with other Devices



- ▲ The CardioPulse Go may be connected/combined with other medical devices such as:
 - ECG:
 - Patient cables and electrodes
 - ECG suction pump
 - Thermal paper
 - CARDIOVIT CS-104
 - Trolley
- ▲ Accessory equipment connected to the analogue and digital interfaces must be certified according to the respective IEC standards (e.g. IEC/EN 60950 for data processing equipment and IEC/EN 60601-1 for medical equipment). Furthermore, all configurations shall comply with the valid version of IEC/EN 60601-1. Everybody who connects additional equipment to the signal input part or signal output part configures a medical system and is therefore responsible that the system complies with the requirements of the valid version of IEC/EN 60601-1. If in doubt, contact the technical service department or your local representative.
- ▲ Any other equipment used with the patient must use the same common earth as the CardioPulse Go.
- ▲ Special care must be exercised when the unit is used with high-frequency equipment. Use the original Spacelabs Healthcare patient cable to avoid possible signal interference during ECG acquisition and to reduce the risk of burns in case of a lack of potential equalisation. However, the stimulation units should only be used at a sufficient distance from the electrodes and both devices must be connected to the same potential equalisation. If in doubt, the patient should be disconnected from the device.
- ▲ This device can safely be used with pacemaker patients.
- ▲ There is no danger when using this device simultaneously with electrical stimulation equipment.
- ▲ If the device is part of a medical system, only the original Spacelabs Healthcare patient cable must be used with, and connected to, the CardioPulse Go.
- ▲ If the patient cable should become defective after defibrillation, a lead-off indication is displayed on the screen (see page 40).
- ▲ Portable communication devices, HF radios and devices labeled with the  symbol (non-ionic electromagnetic radiation) can affect the operation of this device (page 99).

1.8 Network safety



- ▲ When the CardioPulse Go is part of a *network (LAN, WLAN, HIS, etc.)*, the operator of the network/data coupling must take appropriate security measures to protect the transmission of data. Networks that are not protected and maintained can lead to failure of the data transmission or to incorrect transmission of data, which in turn can result in danger to the patient. For further safety notes, see chapter 9.

1.9 Maintenance



- ▲ Danger of electric shock! Do not open the device. There are no serviceable parts inside. Refer servicing to qualified technicians authorized by Spacelabs Healthcare only.
- ▲ Before cleaning and to isolate the AC power supply, switch the unit off and disconnect it from the AC power by removing the plug.
- ▲ Do not use high-temperature sterilization processes (such as autoclaving). Do not use E-beam or gamma radiation sterilization.
- ▲ Do not use aggressive or abrasive cleaners.
- ▲ Do not, under any circumstances, immerse the device or cable assemblies in liquid.

1.10 Serious incidents



- ▲ If any serious incident occurs in relation to the CardioPulse Go, such incident needs to be reported to Spacelabs Healthcare as well as the competent national authority of the country in which the user and/or patient is established.

1.11 Terms of Warranty

Your CardioPulse Go is warranted against defects in material and manufacture, as stated in the Terms and Conditions. Excluded from this warranty is damage caused by an accident or as a result of improper handling. The warranty entitles to free replacement of the defective part. Any liability for subsequent damage is excluded. The warranty is void if unauthorized or unqualified persons attempt to make repairs.

In case the device is defective, send it to your local Spacelabs Healthcare representative or directly to the manufacturer. The manufacturer can only be held responsible for the safety, reliability and performance of the device if:

- assembly operations, extensions, readjustments, modifications or repairs are carried out by persons authorized by him, and
- the Spacelabs Healthcare device and approved attached equipment is used in accordance with the manufacturer's instructions, and.
- the maintenance intervals as stated in the section [Maintenance](#) are observed.



There are no express or implied warranties which extend beyond the warranties hereinabove set forth. Spacelabs Healthcare makes no warranty of merchantability or fitness for a particular purpose with respect to the product or parts thereof.

Spacelabs Healthcare assumes no liability for the loss of data saved on the computer or on the device. The owner is solely responsible for the data backup.

1.11.1 Additional statements

FCC statement

This equipment has been tested and found to comply with the limits for a class A digital device, pursuant to both Part 15 of the FCC (Federal Communications Commission) Rules and the radio interference regulations of the Canadian Department of Communications. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with this instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

This device contains FCC ID: **Z64-WL18DBMOD**

When using the WiFi networking option, operation of this equipment requires the prior coordination with a frequency coordinator designated by the FCC for the Wireless Medical Telemetry Service. This device complies with Part 15 of the FCC rules. Operation is subject to the following conditions:

- This device may not cause harmful interference.
- This device must accept any interference received, including interference that may cause undesired operation.



- ▲ Any changes or modifications to this equipment not expressly approved by Spacelabs Healthcare may cause harmful radio frequency interference and void your authority to operate this equipment.
- ▲ Within the 5150 to 5250 MHz band (5 GHz radio channels 34 to 48) the module type cB-0941 is restricted to indoor operations to reduce any potential for harmful interference to co-channel MSS operation.

1.12 Safety Symbols and Pictograms

1.12.1 Symbols used in this document

The safety level is classified according to ISO 3864-2. The following overview shows the safety symbols and pictograms used in this manual.



For a direct danger which could lead to severe personal injury or to death.



For a possibly dangerous situation which could lead to severe personal injury or to death.



For a possibly dangerous situation which could lead to personal injury. This symbol is also used to indicate possible damage to property.



For general safety notes as listed in this chapter.



For electrical hazards, warnings or precautionary measures when dealing with electricity.



Note For possibly dangerous situations which could lead to damages to property or system failure. **Important** or helpful user information.

Reference to other instructions.

Touch-sensitive areas

This symbol is used to designate touch-sensitive areas that might not be self-evident.



Touch (to open/close menus and perform functions) CardioPulse Go



Move up or down.



Move to the right or left



Scroll up or down using two fingers

1.12.2 Symbols used on the device

General used symbols see [14 Appendix - Symbols](#).

Note: All other symbol used for operating elements and connectors are described in chapter [2.3](#), [2.4](#) and [2.5](#)



Consulting of the user guide is mandatory before using the CardioPulse Go.



Attention: consult accompanying documents.



Warning: Projectile Hazard. Keep outside MRI scanner.



Potential equalisation.



CF symbol. The device is classified safe for internal and external use. However, it is only defibrillation protected when used with the original Spacelabs Healthcare patient cable.

IP20

The device is protected against foreign body diameter greater than >12.5 mm. No water protection. Only for indoor use!

R_xonly

Prescription Only. Federal law restricts this device to sale by or on the order of a physician.



Symbol for the recognition of electrical and electronic equipment

Equipment/components and accessories no longer required must be disposed of in a municipally approved collection point or recycling center. Alternatively, you can return the equipment to your supplier or the manufacturer for disposal. Improper disposal can harm the environment and human health.



Attention: non ionizing electromagnetic radiation. The device contains an HF transmitter (Wi-Fi).

The CardioPulse Go radiates high-frequency electromagnetic energy and can disturb other devices if not installed and operated in accordance with the user guide. CardioPulse Go However, there is no guarantee that no interference can occur in certain installations. If the CardioPulse Go causes interferences, these can be determined by switching the device off/on or by transmitting/not transmitting ECG data. The user can take the following measures to solve this problem:

- Increase the distance between the disturbed device and the CardioPulse Go. A minimum distance of 20 cm must be kept between the device and a pacemaker.
- Turn the device to change the angle of radiation.
- Connect the device to a different AC power connector.


For more details, see [page 97](#).

2 Introduction

The CardioPulse Go is a 12-channel ECG unit designed to record, display, and measure resting ECGs. The multi-touch screen allows easy and intuitive operation to efficiently enter patient data, record ECGs and adjust device settings.

The CardioPulse Go has the following features:

2.1 Main components of the CardioPulse Go

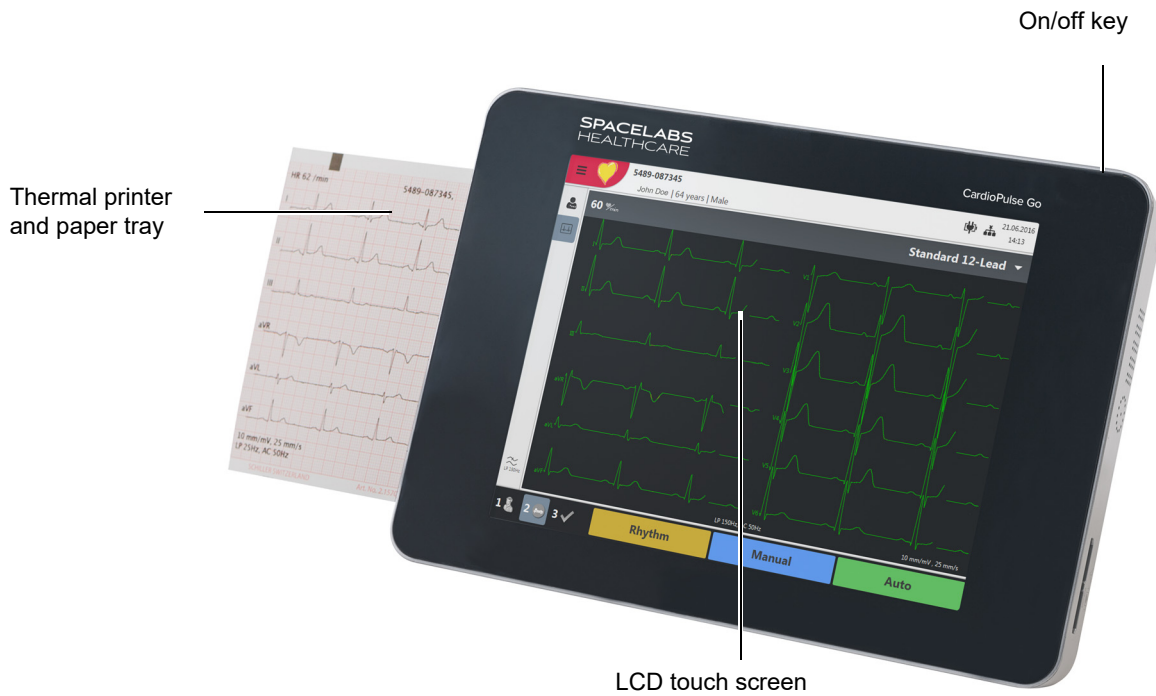

2.1.1 Standard

- Pacemaker detection
- Manual rhythm printout in real time (leads, speed and amplitude can be changed)
- Auto mode recording (10 seconds) with user-defined layout
- Resting-Rhythm
- Measurements
- Full disclosure of all 12 channels
- Display of reversed electrodes
- Recording review
- Connectivity
 - Wi-Fi
 - LAN
- Schiller Link

2.1.2 Options

- Interpretation
- Worklist

2.2 Connections

- RJ-45 Ethernet connector (network)
- 2 USB interfaces for software updates with a USB stick and connection of a Bar-code reader
- USB type B interface (is not intended for the US market)
- Kensington lock

2.3 Display

The display will vary according to the task being carried out. In all screens, however, the top, bottom and right areas always display the same category of information. Example for a typical patient data view:

The screenshot displays the 'Entering Patient Data' screen. At the top, there is a navigation bar with a menu icon, a heart icon, and the text 'Patient ID / Visit ID' followed by 'John Doe 53 years | Male'. To the right of the navigation bar are status icons for Worklist item, Printer jobs, Export status, Battery/AC power, Network status, and Storing capacity, along with the date and time '05.04.2019 08:40'. Below the navigation bar is a form for entering patient data with fields for Patient ID, Visit ID, First name, Last name, DOB, Gender (set to Male), Height [cm], Weight [kg], Ethnicity (set to Caucasian), and Pacemaker (set to Unknown). A virtual keyboard is visible below the form. At the bottom of the screen, there are three numbered steps: 1 (patient icon), 2 (arrow icon), and 3 (checkmark icon). A green 'Record ECG' button is located at the bottom right. A 'Review' button is also shown at the bottom right. A sub-menu is open over the keyboard, listing 'Patient data' and 'Recording data'. Various callouts point to different parts of the interface, such as 'Access to the main menu', 'Entering Patient Data', 'Display of patient data', 'Worklist item', 'Printer jobs', 'Export status', 'Battery/AC power', 'Network status', 'Storing capacity', 'Date and time', 'Sub-menu', 'Access to process steps', 'Alphanumeric or just numeric virtual keyboard', and 'Function key, go to the next step. The function keys change their function depending on the selected view.'

Access to the main menu

- Worklist
- Recorder
- Memory
- Settings ⚙️
- Maintenance 🔧

Entering Patient Data

Display of patient data

Worklist item

Printer jobs

Export status

Battery/AC power

Network status

Storing capacity

Date and time

05.04.2019 08:40

Patient ID / Visit ID
John Doe 53 years | Male

Sub-menu

- Patient data
- Recording data

Access to process steps

- 1 Enter patient data
- 2 Apply electrodes, check and record resting ECG
- 3 Check and save recording

Alphanumeric or just numeric virtual keyboard

Function key, go to the next step. The function keys change their function depending on the selected view.

Record ECG

Review


Function key to return to the review screen; this is only available when the recording has not yet been accepted.

2.4 Multi-touch screen operation

The touch-sensitive areas of the screen are located as follows:

The diagram illustrates three states of the CardioPulse Go interface with callouts for touch-sensitive areas:

- Top Screenshot:** Shows the 'Patient ID / Visit ID' form. A callout '1' points to the main menu icon (three horizontal lines) in the top left. Another callout points to the 'Patient ID' and 'Visit ID' input fields, with the text: 'Enter patient and recording data, or import patient and recording data from an EMR system using the symbol [envelope icon]'. A third callout points to the 'Record ECG' button at the bottom right.
- Middle Screenshot:** Shows the 'Main menu' overlay. A callout points to the 'Recorder' option, with the text: 'Select a menu option'. Another callout points to the 'Memory' option.
- Bottom Screenshot:** Shows the 'Patient ID / Visit ID' form with a sidebar menu. A callout points to the 'Patient data' and 'Recording data' sections, with the text: 'Show/hide menu texts'. Another callout points to the 'Reset' and 'User previous patient' options at the bottom of the sidebar.

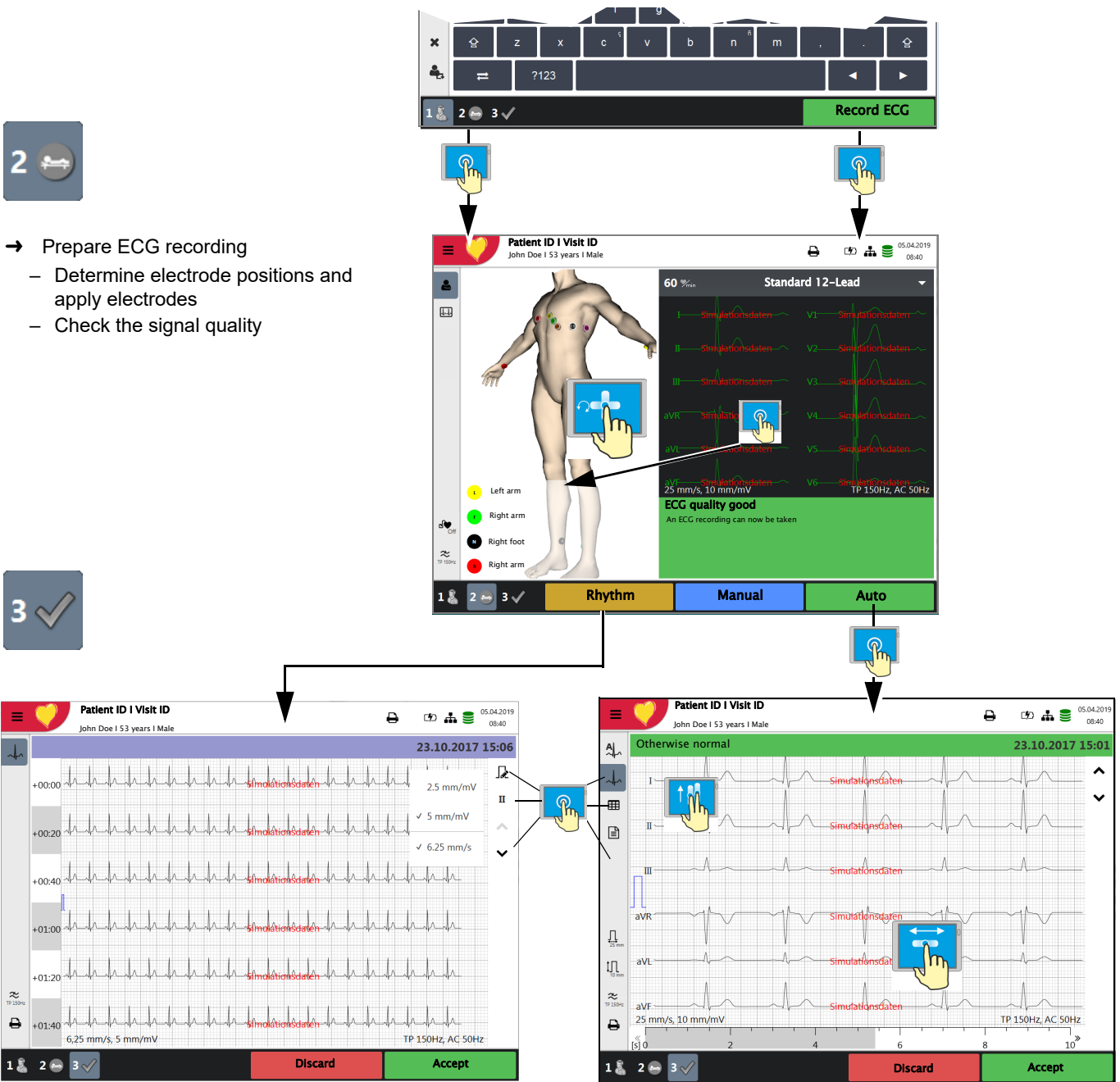


REF: 2.511461 REV B / CAT: 070-3062-00 REV B

2.5 Connection panel

CAUTION


- ▲ All externally connected hardware must be approved by Spacelabs Healthcare. Connection of any hardware not approved by Spacelabs Healthcare is at the owner's risk. Moreover, the unit's warranty may become invalid.

2.5.1 Back panel

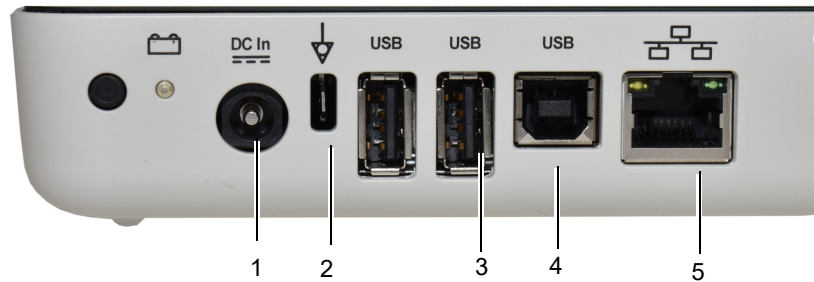


- (1) Power input 15 VDC
- (2) Potential equalisation stud. The potential equalisation stud is used to equalise the ground potential of the unit to that of any nearby AC-powered equipment. Use the hospital or building common ground for all AC-powered units.
- (3) USB interfaces for the Barcode scanner and USB memory sticks.
- (4) USB type B interface is not intended for the US market
- (5) RJ-45 Ethernet LAN connection (Local Area Network)

2.5.2 Right hand side panel

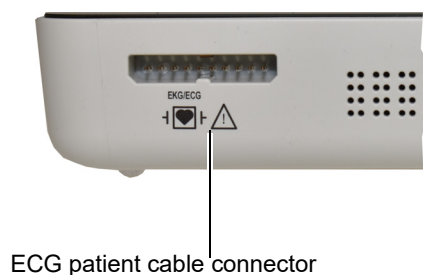

ECG patient cable connector

CAUTION

- ▲ The patient cable as well as the connector comply with the safety standard CF , e.g. they are fully floating and isolated and defibrillation protected.
- ▲ The unit is only CF rated and defibrillation protected if used with the original patient cable see [11.3.1 ECG CablePatient cables](#).

3 Operation

3.1 Start-up and Preparation



- ▲ Danger of electrical shock. Do not operate the unit if the earth connection is suspect or if the power supply unit/AC power lead is damaged or suspected of being damaged.

3.1.1 Location

- Do not keep or operate the unit in a wet, moist or dusty environment. Avoid exposure to direct sunlight or heat from other sources.
- Do not allow the unit to come into contact with acidic vapors or liquids.
- The CardioPulse Go must not be placed in the vicinity of X-ray or diathermy units, large transformers or electric motors.

3.1.2 Connection of external cable assemblies and ancillary equipment



1. Connect the external power supply unit to the AC power.
2. Connect the external power supply unit's output to the back panel. The AC power indicator lamp is lit.
3. Leave the CardioPulse Go connected to the AC power for 8 hours to fully charge the battery (see [page 23](#)).
4. Connect the potential equalisation cable.
5. Connect the patient cable (side panel).
6. Connect any ancillary and optional equipment (see [page 20](#)). These may include the following:
 - Network cable
 - USB Barcode reader

3.1.3 Potential equalisation



The potential equalisation stud at the back of the unit is used to equalise the ground potential of the CardioPulse Go to that of all AC-powered equipment in the vicinity. Use the hospital or building common ground. A yellow/green ground cable is supplied as an option (CAT no.: 161-0280-00).



- ▲ Danger of triggering ventricular fibrillation! If the CardioPulse Go is used together with devices that are designed for direct cardiac application, both devices must be connected to the hospital/building common ground (potential equalisation) to prevent equalizing currents between different device potentials.

3.2 Switching on / off

→ The unit is switched on and off with the **On / Off** key.



3.2.1 Login and Logout / Emergency ECG



▲ To prevent non-authorized users to access patient information, manipulate settings or the software, we strongly recommend changing the default password to a strong password according to the standard rules. ([section 8.5.1 Setting user name and password for access control, page 82](#))

If Access control is activated, the following dialogs are displayed:

Login

→ Enter the user name (administrator) and password (administrator) and press “Login” to log in.

Emergency ECG

→ Select “**Emergency ECG**” to bypass the login procedure and to perform an emergency ECG.
→ You are automatically logged off when the ECG recording is accepted.

Logout

→ Open the main menu  and select .



- If **Emergency ECG** is selected, the login is bypassed for one acquisition. The Login screen appears again after accepting the emergency ECG recording. No other workflow is accessible.
- User roles and privileges are assigned to individual users and that can affect access to a workflow area and the functions that can be carried out. If a function is not available, it means that the user logged in does not have the privileges required. Individual users, and the user groups and privileges defined for individual users are defined by the administrator.



3.3 Power supply





3.3.1 AC power and battery indicators



The unit can either be operated from the AC power supply or from the built-in rechargeable battery. Battery charging is indicated by the LED below the battery symbol.

The current power source is displayed in the top right corner of the screen when the unit is switched on:

- AC power via external AC power supply unit  , battery is being charged
- Internal rechargeable battery ()
- When running on battery power and the battery capacity is limited, the battery symbol is flashing.
- When AC power is connected and the battery is charging, the battery symbol is displayed 'filling'.

-  Full
-  Half full
-  Almost empty
-  Empty


Battery capacity

The internal battery provides power for up to four hours. When the unit is running on battery power (AC power not connected), the battery symbol indicates the battery status. When the battery is full, the symbol is solid.

When running on battery power and the battery capacity is low, the battery symbol turns red.

Battery charging

The battery is charged when the unit is connected to the AC power supply. The unit can remain connected to the AC power supply without damage to either the battery or the unit.

When the battery is not fully charged and the AC power supply is connected, the battery LED  is blinking, indicating that the battery is being charged.

3.3.2 Isolating the device from the AC power supply

To isolate the device from the AC power supply, remove the AC power supply plug from the external power supply unit.

3.4 System and ECG settings



- The system settings (time, date, device ID etc.) and other general settings are described on [page 72](#).
- Resting ECG settings (auto format, user defined leads, print options, interpretation, rhythm lead definition, etc.) are described on [page 52](#).

Overview **Menu > Settings**

Menu Settings	Sub-menu
ECG (Page 72)	<ul style="list-style-type: none"> • Leads & cable • Filters & formulas • Views & layouts • Additional leads • Interpretation • Rhythm ECG
Reports (Page 75)	<ul style="list-style-type: none"> • General • Manual printout • Resting ECG • Rhythm ECG • EMR integration
Connectivity (Page 78)	<ul style="list-style-type: none"> • Update server • Ethernet • WLAN
Regional (Page 79)	<ul style="list-style-type: none"> • Date & time • Keyboard • Language • Units • Patient ID system
General (Page 80)	<ul style="list-style-type: none"> • Info • Power management • Station • Update • Manage licenses • Mandatory fields • Workflow • Access control • Printer

3.5 Changing the printing paper



Important

The device is delivered without printing paper inserted. The thermal paper is sensitive to heat, humidity and chemical vapors. The following points apply to both paper storage and archiving of recordings:


- Before use, keep the paper in its original cardboard cover. Do not remove the cardboard cover until the paper is to be used.
- Store the paper in a cool, dark and dry location.
- Do not store near chemicals, e.g. sterilization liquids.
- Do not store in a plastic cover.
- Certain glues can react with the paper. Therefore, do not use glue to attach the printout onto a mounting sheet.

Good printouts can only be achieved when original chart paper or chart paper of the same quality is used.


If different paper is used:

- The operator is solely responsible for making sure that the printing quality is flawless.
- The printing quality needs to be checked before the device is used for patient recordings


1. Pull out the paper tray.

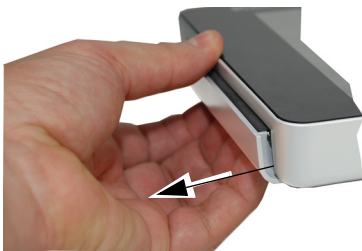

2. Remove the remaining paper.

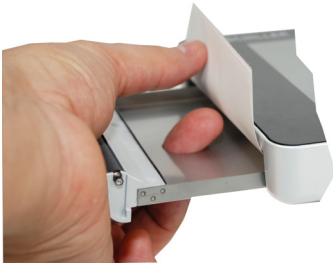

3. Place a new paper pack into the paper tray with the printed (grid) side facing upwards and the black paper mark facing the top of the unit.

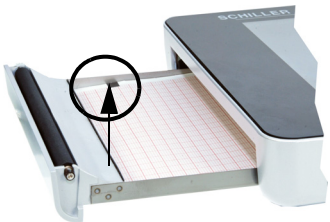


4. Pull out the first page as shown on the left.

5. Push the paper tray home until it locks into place.




3.6 Patient / recording data

In the patient data screen, new patients can be entered and previously stored patient data can be edited.

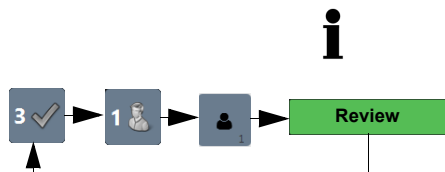

If a recording is performed without having entered a patient or visit ID, a UUID is generated instead of a patient ID, "Emergency ECG" is given instead of a last name, and the date and time are given instead of a first name. If you want to enter patient data once the recording has been performed (and before it has been accepted), you can jump to the Patient data screen, enter the data and use the Review key to jump back to Step 3 to accept (save) the recording.

With the data of the current patient, you can:

- edit it directly in the entry fields
- obtain the data from the server by entering the Patient or Visit ID (configuration: see page 78)
- press **X** to reset the data and enter a new patient
- press  to use the previous patient data
- read the Patient ID with a Barcode scanner.
 - Use the alphanumeric keyboard (shown) to enter the patient data.
 - For numeric entry fields, only the numeric keypad is displayed.
 - Press the key **?123/ABC** to display special characters.
 - Use the **Shift** key to switch the keyboard to capitals.
 - To select special characters from the keyboard, **press and hold the key and select the special character.**





Patient data - left entry fields

- Patient ID** Enter the patient's identification number.
- Last name** Enter patient's name (maximum 50 characters).
- First name** Enter patient's first name (maximum 50 characters).
- Date of birth** Enter the patient's date of birth in the format dd-mm-yyyy or yyyy.mm.dd.
- Gender** Enter the patient's gender - Male or Female or Undefined.


Patient data - right entry fields



▲ The field **Visit ID** must not be used to enter other types of information (e. g. technician, department). Entering this type of information in the field **Visit ID** may lead to patients being mixed up when the device is connected to the Server.

- Visit ID** The **Visit ID** is a unique patient identification provided by the hospital information system (HIS) (max. 50 characters). For more information on the **Visit ID** and on validation options with regard to the HIS, consult the user guide for the Schiller Server.
- Height** Enter the patient's height.
- Weight** Enter the patient's weight.
- Ethnicity** Select one of the following:
 - Undefined
 - Caucasian
 - Asian
 - Black / African American
 - American Indian / Alaska Native
 - Native Hawaiian / Pacific Islander
 - Hispanic
 - Oriental
 - Other

Pacemaker Select if the patient has a pacemaker (Yes/No/Unknown).

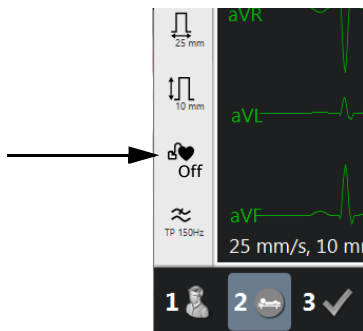

Regardless of this setting, detected pacemaker pulses are indicated in blue and the interpretation states that it is a pacemaker ECG. Visual indication of pacemaker pulses is switched off and needs to be manually activated in the ECG preview.



Recording data - left side

Room

Examination room

Medication

Digitalis

Indication

Reason for medication

Referring physician

Referring physician

Attending physician

Attending physician

Remark

Remarks about the patient/recording



Deleting entered patient data.



The previous patient data is entered again.



All fields (except for Medication) can be set as Mandatory fields: **Menu > Settings > General > Mandatory fields**, see [section 8.5 General, page 80](#). For worklist recordings, defined mandatory fields are not taken into account.

3.6.1 Patient data query (PDQ)

When the unit is connected to Server or another hospital patient database (via network or WLAN), patient data can be filled in automatically when the **Patient ID** or **Visit ID** is entered. This is called **Patient Data Query** or **PDQ**.

Patient data query is possible as follows:

- The patient data is filled in automatically after the Pat ID/visit ID has been entered by the user or read with a Barcode reader (confirm by pressing Enter) (see following).

The PDQ settings are defined in **Menu > Settings > General > Workflow** - the following options are available:

- **Patient data query (PDQ mode)** - select one of the following:
 - Patient ID
 - Visit ID

These settings along with other transmission settings are detailed in the system settings (see [page 70](#)).


PDQ with Barcode reader

If a Barcode reader is attached (see next page), scan the Barcode to enter the **Patient ID / Visit ID**. Patient data is filled in automatically when the **Patient ID/Visit ID** is read with a Barcode reader.



Barcode scanner configuration: see document 2.510721.

3.6.2 PDQ in the worklist

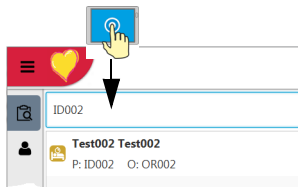

If you use the "Worklist" workflow, you can search/enter patient data in the same way (see [page 65](#)).

Tap the Search field and read the **Patient ID** or **Visit ID** using the Barcode reader. The corresponding work item is shown in the worklist.

3.6.3 Barcode reader



A Barcode reader can be attached to the USB port on the back panel to read the Patient ID / Visit ID. Spacelabs Healthcare has tested the following Barcode reader:

→ Symbol Model LS 2208, from Symbol Tech N.Y.

When a Barcode reader is connected, the patient data is read from the Barcode (generated by the hospital system). If an external hospital patient database is available, all patient data is entered in the patient data fields of the CardioPulse Go as described on the previous page.



A worklist can also be downloaded from a hospital patient database giving patient data and specifying the type of recording to be carried out. The worklist is described on [page 65](#).

4 Placing the electrodes



- ▲ Ensure that neither the patient nor the leading parts of the patient connection nor the electrodes (including the neutral electrodes) come in contact with other persons or conductive objects, even when these are earthed.

4.1 Basics

Careful application of the electrodes and good electrode contact is important for a good recording (see electrode positioning on pages 33 - 40).

A minimal resistance between skin and electrode is required to obtain the best ECG signal and ensure the highest quality ECG recording. Therefore, please note the following points:

1. Only use electrodes that are recommended by Spacelabs Healthcare (see accessories)
2. Before using disposable electrodes, check that the expiration date has not yet passed.
3. To increase the electrode's conductivity and adherence:
 - Shave the areas where the electrodes are to be placed, if necessary.
 - Thoroughly clean the areas with alcohol or soapy water.
 - Let the skin dry before applying the electrodes.
 - ¹When applying the electrodes, ensure that a layer of gel is between the electrode and the skin.
4. Check the electrode resistance as described in the section 4.10.
5. If the electrode resistance is higher than the acceptable level:
 - Remove the electrode and use an abrasive cleaning pad or abrasive cleaning gel ² to remove the uppermost layer of epidermis.
 - Apply the electrode. Always use a new disposable electrode.
6. Ensure that the patient is warm and relaxed before you start the recording.
7. After the recording, remove the electrodes. Clean the suction or vacuum electrodes according to the manufacturer's instructions.

-
1. Electrode gel is integral with single-use electrodes and extra gel does not need to be applied when single-use electrodes are used. For biotab electrodes, solid conductive gel is incorporated in the adhesive.
 2. Dedicated abrasive cleaning gel gives very good results in reducing the skin-electrode resistance.

4.2 Electrode Identification and Color Code


The electrode color codes in the following sections correspond to Code 1 (IEC) for the graphics and to Code 2 (AHA) in the tables

	IEC		AHA	
	IEC label	Color	AHA label	Color
Limb	R	Red	RA	White
	L	Yellow	LA	Black
	F	Green	LL	Red
Chest according to Wilson	C1	White/red	V1	Brown/red
	C2	White/yellow	V2	Brown/yellow
	C3	White/green	V3	Brown/green
	C4	White/brown	V4	Brown/blue
	C5	White/black	V5	Brown/orange
	C6	White/violet	V6	Brown/violet
Neutral	N	Black	RL	Green



The patient cable (type IEC or AHA) is set in the menu [Leads & cable, see chapter 5.6.1](#).

4.3 Standard 10-lead resting ECG

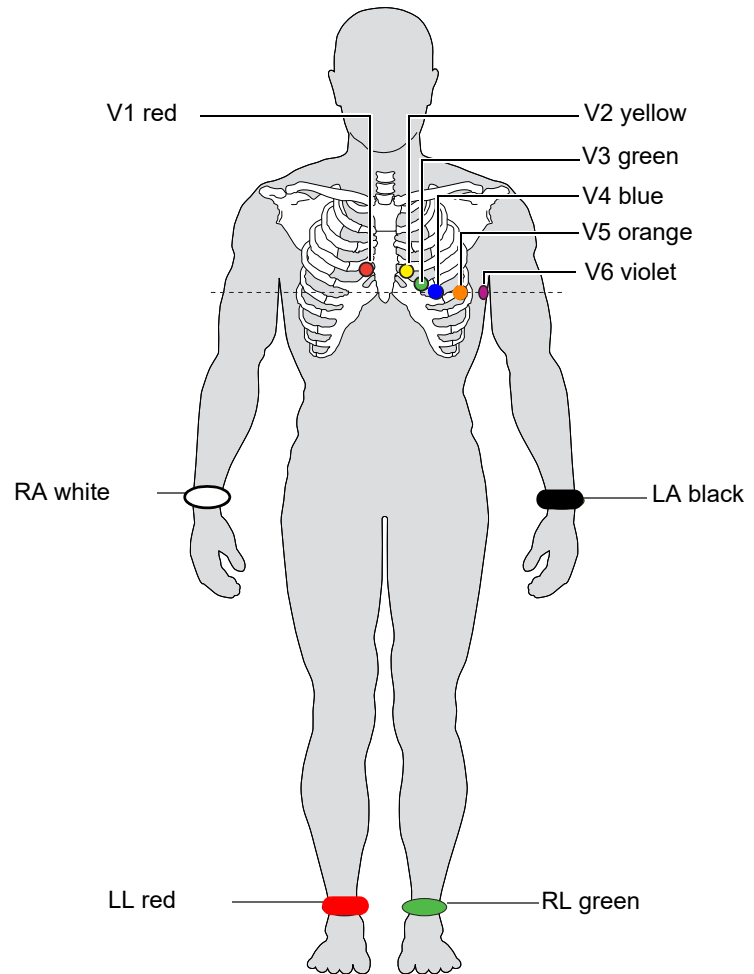


4.3.1 Electrode placement for standard leads

AHA label	Electrode placement
V1 red	→ Fourth intercostal space at the right sternal border
V2 yellow	→ Fourth intercostal space at the left sternal border
V3 green	→ Midway between C2 and C4
V4 blue	→ Fifth intercostal space on the mid-clavicular line
V5 orange	→ Anterior axillary line on the same horizontal level as C4
V6 purple	→ Mid-axillary line on the same horizontal level as C4
LA black	→ Left arm (resting ECG)
RA white	→ Right arm (resting ECG)
LL red	→ Left foot (resting ECG)
RL green	→ Right foot (resting ECG)

The electrode resistance can be checked in the electrode test screen ([see page 40](#)).


4.4 Balanced

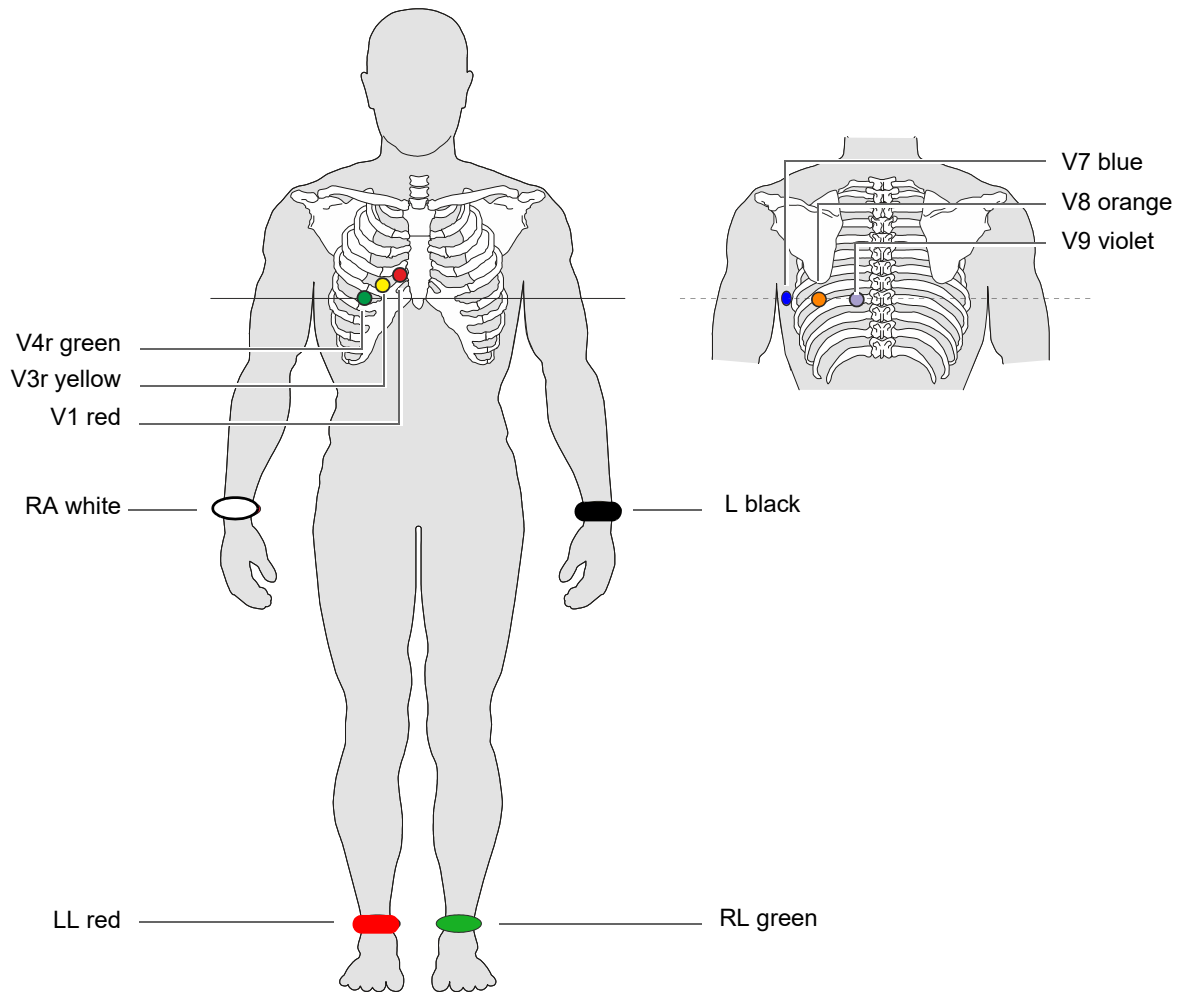

Balanced, 10-wire cable

AHA label	Electrode placement
V1 brown / red	→ Fourth intercostal space at the right sternal border.
V3r brown / yellow	→ Left of the mid-scapular line at the level of C3
V4r brown / green	→ Left of the mid-scapular line at the level of C4
V7 brown / blue	→ Left axillary line at the level of C4.
V8 brown / orange	→ left posterior axillary line opposite C4
V9 brown / violet	→ Left axillary line at the level of C4, opposite C3
LA black	→ Left arm
RA white	→ Right arm
LL red	→ Left foot
RL green	→ Right foot


4.5 Pediatric

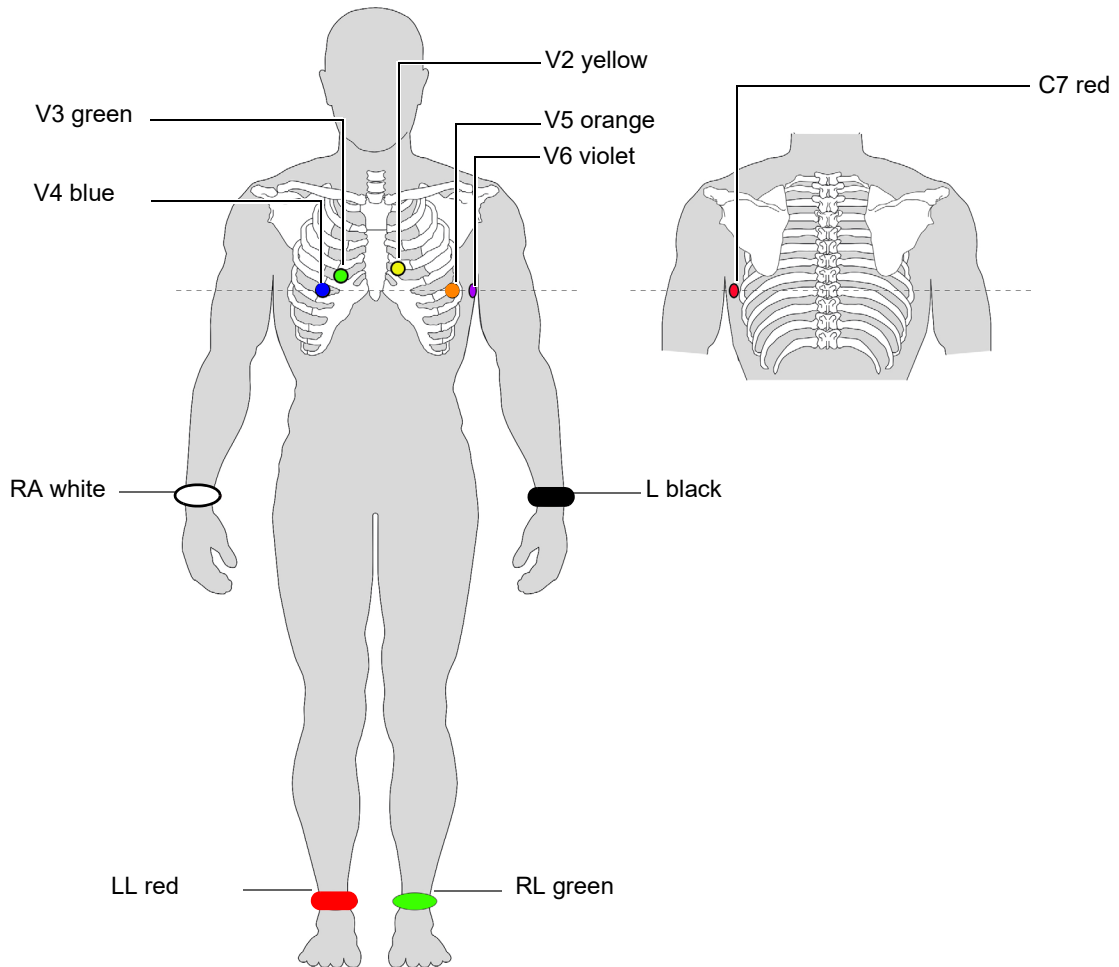

AHA label	Electrode placement
V4 brown / blue	→ Fifth intercostal space on the mid-clavicular line.
V3 brown / green	→ Fourth intercostal space, above C4r
V2 brown / yellow	→ Fourth intercostal space at the left sternal border
V5 brown / orange	→ Anterior axillary line on the same horizontal level as C4
V6 brown / violet	→ Mid-axillary line on the same horizontal level as C4
V7 (V1 brown / red)	→ Left posterior axillary line at the level of C4.
LA black	→ Left arm (resting ECG)
RA white	→ Right arm (resting ECG)
LL red	→ Left foot (resting ECG)
RL green	→ Right foot (resting ECG)

4.6 Right precordials (V3r-V6r)


Since the treatment of an infarction might depend on the influence of the right ventricle, it is suggested to perform additional recordings with right precordial leads in the case of an acute infarction of the right ventricle's inferior wall (Circulation 2007).

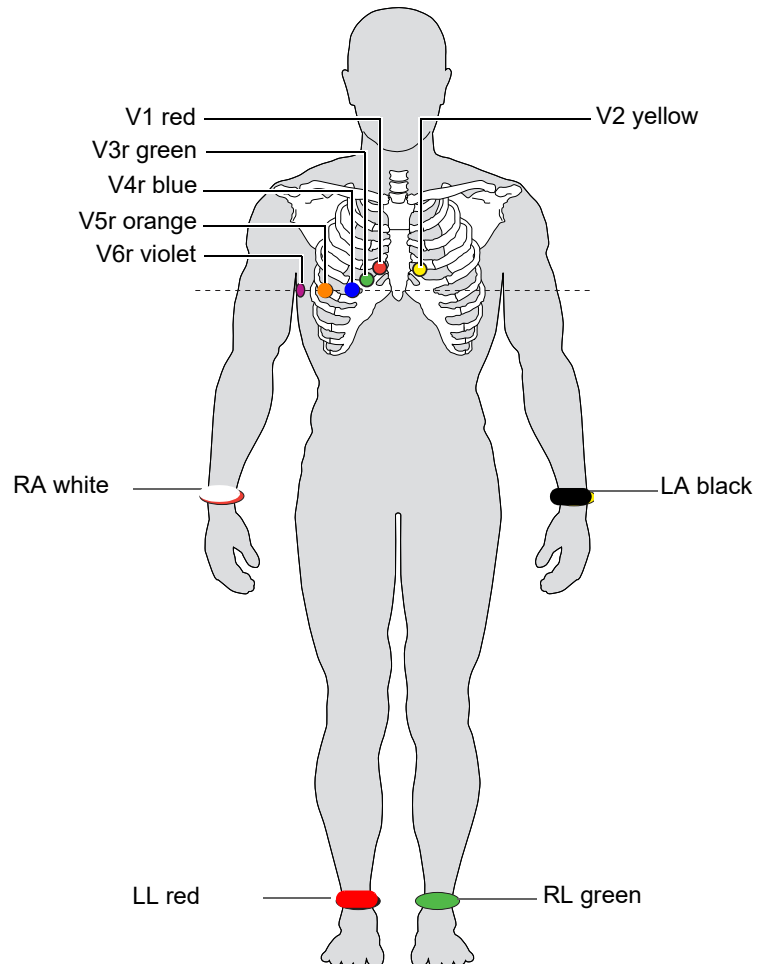

AHA label	Positioning
V1 brown / red	→ Fourth intercostal space at the right sternal border.
V2 brown / yellow	→ Fourth intercostal space at the left sternal border.
V3 brown / green	→ Designated point halfway between V1 and V4r.
V4 brown / blue	→ Fifth intercostal space on the mid-clavicular line.
V5 brown / orange	→ Anterior axillary line on the same horizontal level as V4r.
V6 brown / violet	→ Mid-axillary line on the same horizontal level as V4r.
LA black	→ Left arm
RA white	→ Right arm
LL red	→ Left foot
RL green	→ Right foot

4.7 Standard (V4r)


ACC/AHA guidelines recommend examining patients suffering from a myocardial infarction with inferior ST elevation for possible RV ischemia or RV infarction; this examination should be performed with a right precordial V4r lead.

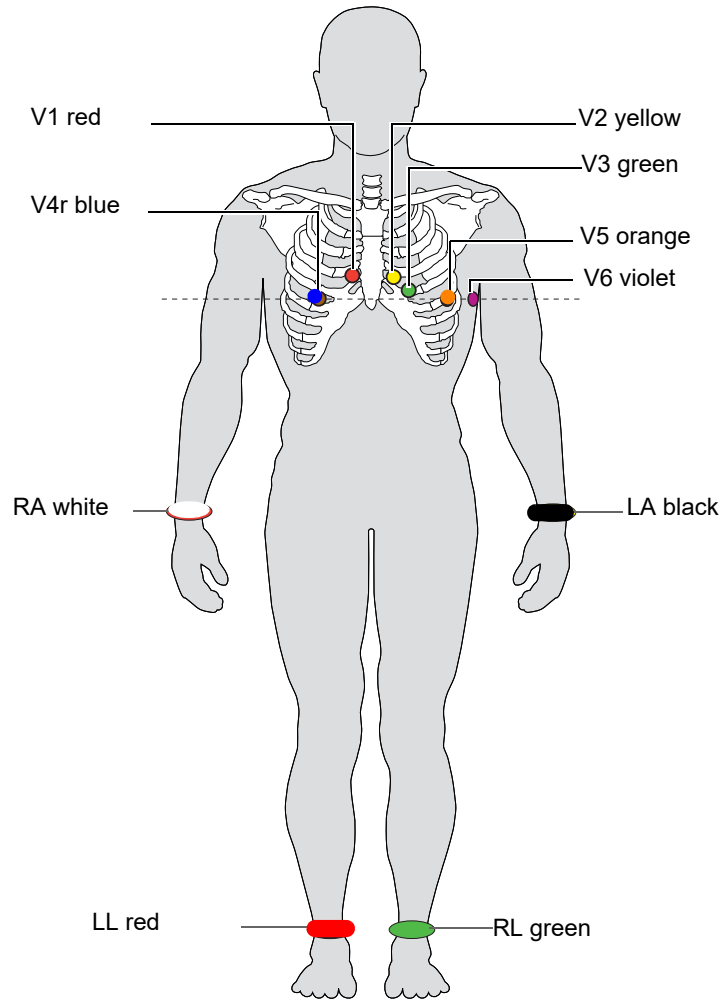

AHA Label	Electrode placement
V1 brown / red	→ Fourth intercostal space at the right sternal border.
V2 brown / yellow	→ Fourth intercostal space at the left sternal border.
V3 brown / green	→ Midway between V2 and V4.
V4 brown / blue	→ Fifth intercostal space on the mid-clavicular line.
V5 brown / orange	→ Anterior axillary line on the same horizontal level as V4.
V6 brown / violet	→ Mid-axillary line on the same horizontal level as V4.
LA black	→ Left arm
RA white	→ Right arm
LL red	→ Left foot
RL green	→ Right foot

4.8 Left posterior V7-V9



If an acute coronary occlusion is strongly suspected, it is recommended to also register posterior chest wall leads V7 to V9)

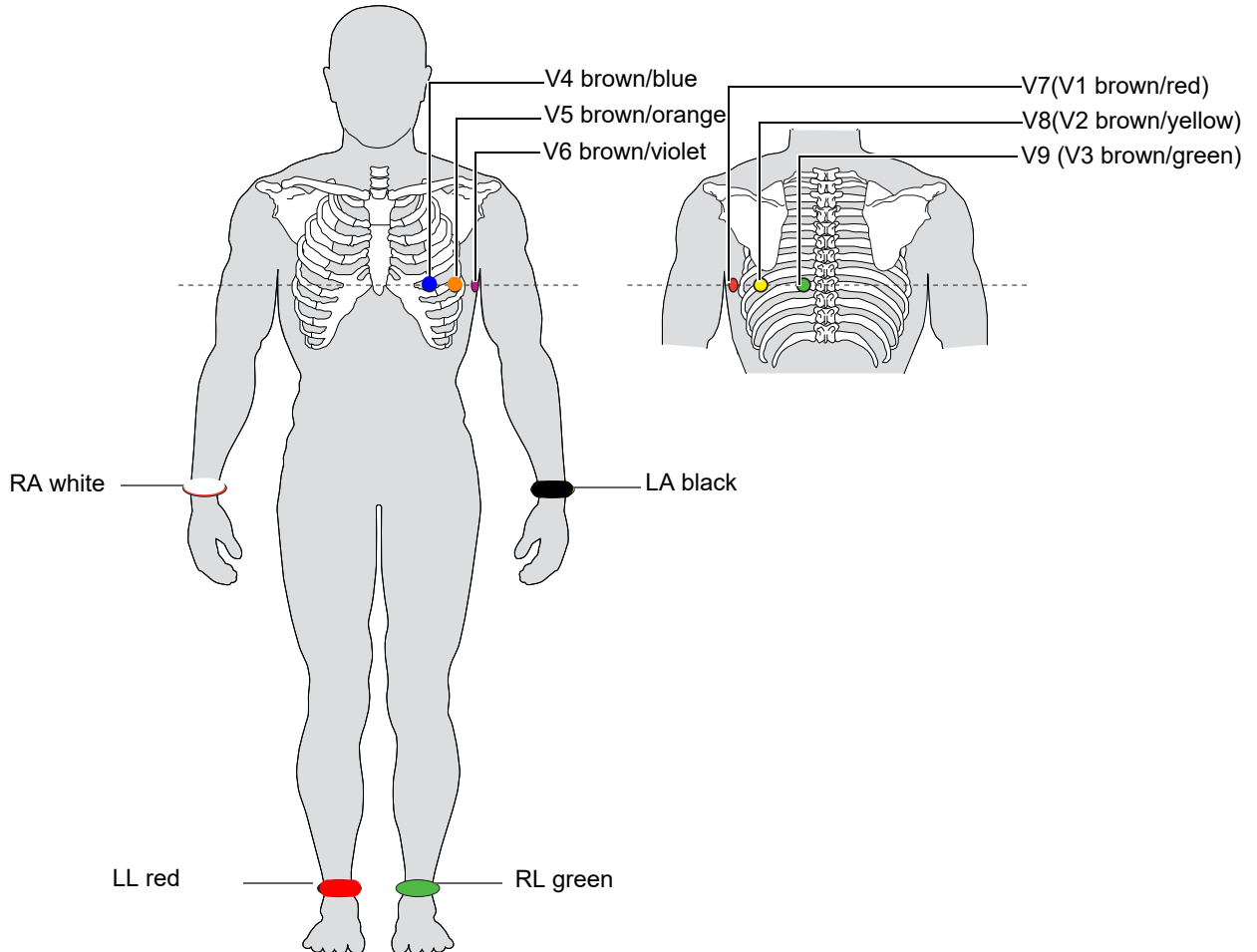

AHA label	Positioning
V7 (V1 brown / red)	→ Left posterior axillary line at the level of V4.
V8 (V2 brown / yellow)	→ Left of the mid-scapular line at the level of V4.
V9 (V3 brown / green)	→ Left paravertebral line at the level of V4.
V4 brown / blue	→ Fifth intercostal space on the mid-clavicular line.
V5 brown / orange	→ Anterior axillary line on the same horizontal level as V4.
V6 brown / violet	→ Mid-axillary line on the same horizontal level as V4.
LA black	→ Left arm
RA white	→ Right arm
LL red	→ Left foot
RL green	→ Right foot

4.9 Nehb leads

The Nehb leads are bipolar chest leads. They are of special interest for the diagnosis of changes in the posterior ventricle wall. Three leads are arranged in the form of a triangle, also called the “small cardiac triangle”. Nehb dorsal (D) is measured between the electrode positions Nax and Nst; Nehb anterior (A) between Nap and Nst, and Nehb inferior (J) between Nap and Nax.

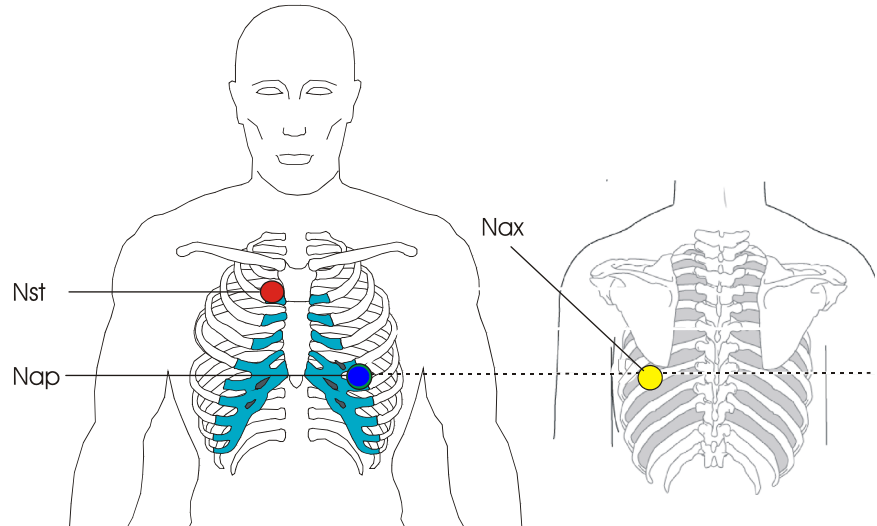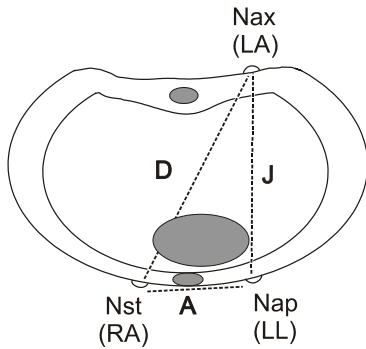

Place the electrodes as follows:

AHA label	Electrode placement
V1 brown / red → Nst : 2nd rib at the right sternal border.	
V2 brown / yellow → Nax : left posterior axillary line (on the back), directly opposite Nap.	
V4 brown / blue → Nap : 5th intercostal space, midclavicular line (cardiac apex).	

Place all other electrodes in the normal positions ([page 33](#)).

4.10 Skin/Electrode Resistance



4.10.1 Electrode and patient cable check

The electrode check is part of step 2 before the start of an ECG recording. The following is checked and displayed:

- Excessive noise (signal noise too high)
 - Due to poor electrode contact
 - AC power interferences (mains filter not activated)
- Electrodes reversed
- Electrodes have come off

The electrode status is shown in the bottom right information field of the screen. If an electrode is displayed red, the suspected cause is displayed. Reapply the electrode.



- If F (LL) or N is not connected or has come off, the electrode resistance cannot be measured and all leads are marked red.

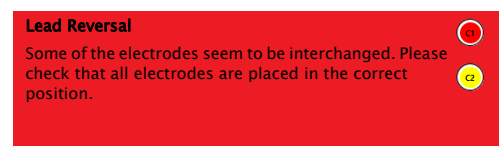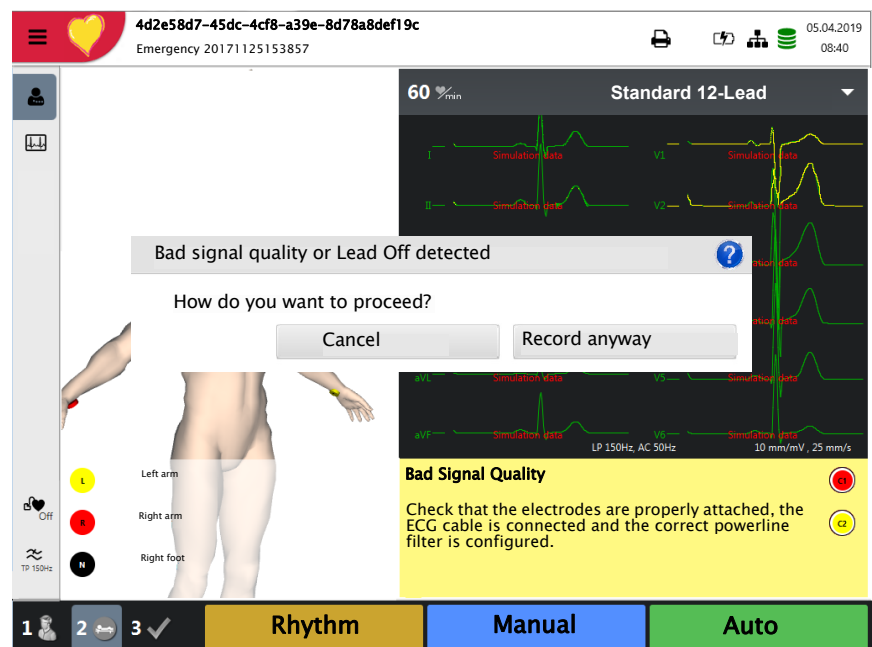

4.11 Lead sequence/lead view


4.11.1 Setting Standard or Cabrera lead sequence

- The lead sequence is defined in the settings.
(Menu > Settings > ECG > Leads & cable).
- In the Lead menu, select between Standard and Cabrera.

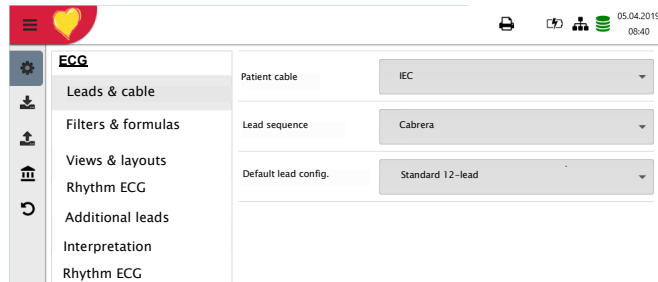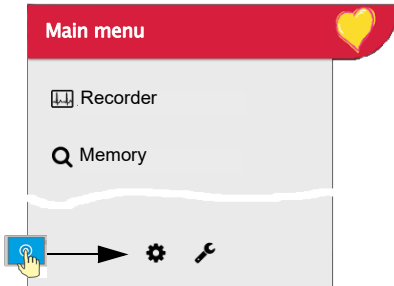

4.11.2 Select the lead view (Standard or other settings)

The lead display can be set directly in the electrode screen in Step 2.

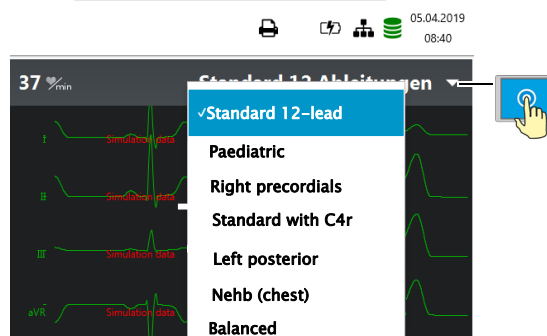

The lead labels on the display and on printouts change accordingly.



Important


Automatic interpretation is only possible when **Standard 12 lead** is set.

5 Resting ECG

WARNING

- ▲ After heavy artifact's or lead off, the displayed heart rate may not be reliable.

CAUTION

- ▲ The safety notes at the beginning of this book must be read and fully understood before taking an ECG recording.
- ▲ The CardioPulse Go device is CF classified . The patient connection is fully isolated. During the ECG recording, ensure that neither the patient nor the leading parts of the patient connection nor the electrodes (including the neutral electrodes) come in contact with other persons or conductive objects, even when these are earthed.
- ▲ Do not operate the unit if the earth connection is suspect or if the AC power lead is damaged or suspected of being damaged.
- ▲ If an external electronic device (e.g. a PC) is connected to the CardioPulse Go, use the potential equalisation stud for earth protection.

i

If another format than the default format is set for the automatic printout, the printout can differ from the format displayed on the screen.


The displayed lead sequence (Standard or Cabrera) can be selected (see [5.6.1 Leads & cable page 52](#)). Amplitude and speed can be defined in the menu [5.6.3 Views & layouts, page 54](#).

For the preview, the following parameters can be freely programmed (before start of the recording):

- Lead configuration
- Filter

Saved recordings can be displayed and printed in another format at any time. For further information on how to define the format, see [5.6.7 Reportspage 58](#).

5.1 Resting ECG - procedural flow diagram

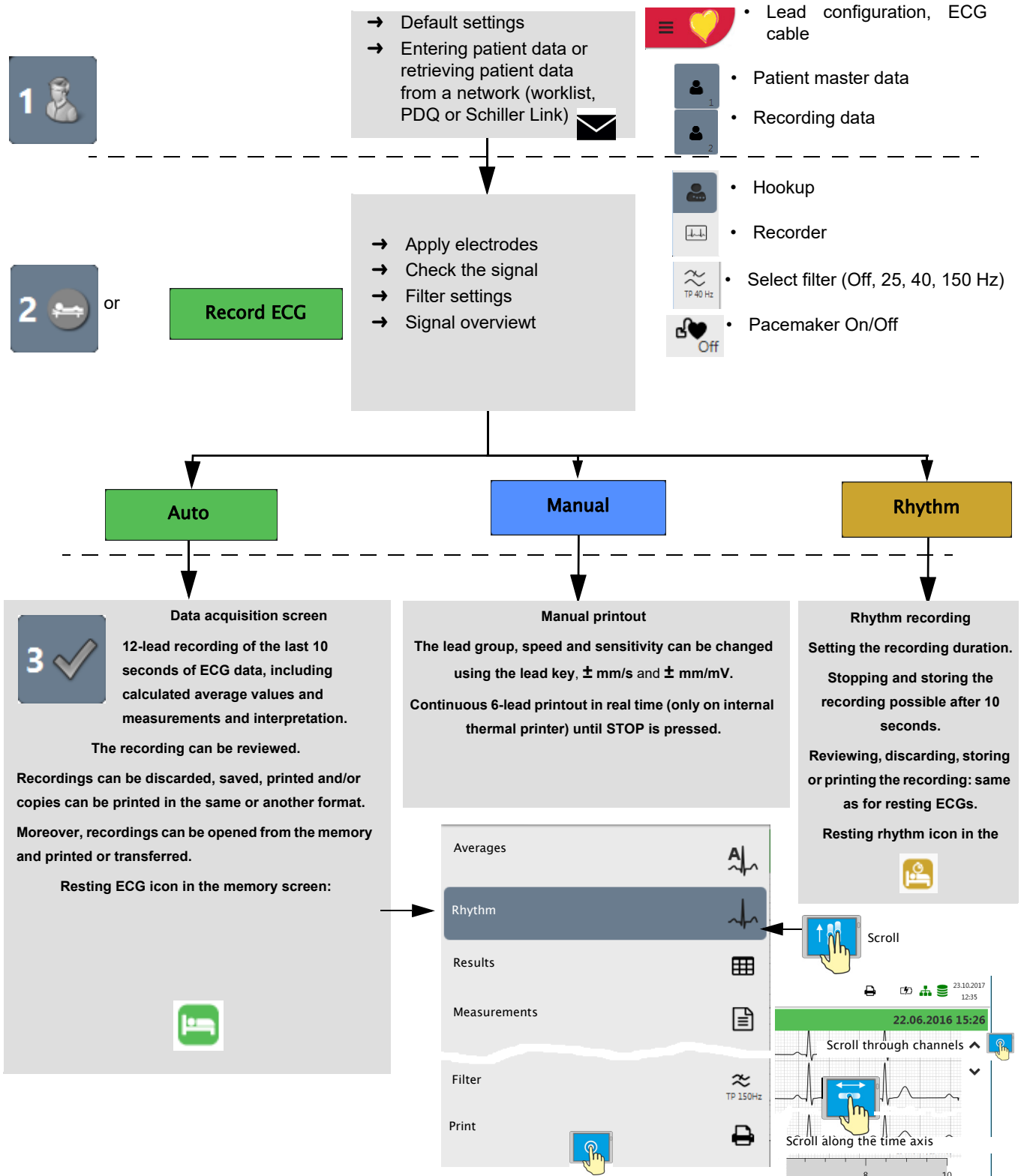

5.1.1 Printing, saving and transferring automatically

Menu > Settings > General > Workflow

Activate **Print after acquisition**, **Transmit after acquisition** and **Delete after transmission** to automatically print and transmit a saved recording or to delete recordings after transmission.



- The transmission settings are detailed in the section Settings (see [page 78](#)).
- Further ECG settings are described later in this section (see [page 52](#)).
- Printing and transfer from the memory is described in the section Memory (see [page 62](#)).
- The settings are saved automatically. The settings can be exported (see [page 71](#)).

5.2 Automatic resting ECG recording

To take an automatic ECG recording, press the **Auto** key. After approx. 10 seconds, the recording is analyzed and the result displayed. The recording can be checked and saved and further printouts can be obtained in different formats. Depending on the setting, the recording is deleted automatically as soon as it has been transmitted, or it remains stored in the memory.

Auto

Amplitude settings (5/10/20 mm/mV) and speed (12.5/25/50 mm/s)

Pacemaker On/Off

Filter Settings

Remaining recording duration

Depending on the settings (see section 5.6.3), only some or none of these pages are displayed.

The grey bar shows the position on the time axis.

Check the recording using (time axis) and (select channels).

→ Select the amplitude and speed for the display .

→ Select filter (Off = 250 Hz, 25, 40 or 150 Hz) .

→ **Accept** the recording.

→ Print the recording .

→ Press **Discard** to exit the preview without storing the ECG.

→ Select a recording for review and printout via the **Memory** menu.

→ Open the menu **Memory** to delete a recording from memory.

5.2.1 Automatic printout

The printout gives the following:

- Heart rate
- Patient ID or Visit ID
- Time and date
- Speed
- Sensitivity
- Filter
- Device ID
- Serial number
- Software version

And any combination of the following (for printout settings, see [page 58](#)):

Patient data

Result

- Interpretation
- Intervals & axis

Measurements

- Detailed measurement table

Rhythm

- ECG recording of all 12 leads in either Standard or Cabrera format (according to selection)

Averages

- Averaged cycles with markings


5.3 Manual rhythm printout

Use this function to print a real-time ECG. The print parameters such as lead sequence, print speed and sensitivity can be changed by the user during the printout.



The real-time ECG is not saved. The chosen settings only apply to the printout.

5.3.1 Starting manual printout

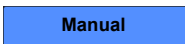

→ To start a manual real-time printout, press the **Manual** key.

The default printout settings are **25 mm/s** and **10 mm/mV**. These settings can be changed, see [section 5.6.7 Reports, page 58](#)



Select lead sequence

→ To change the lead sequence for the printout (Standard I, II, III, aVR, aVL, aVF), press the key **- I - aVF +**.

The Standard and Cabrera lead sequences are as follows:

Lead sequence	Lead group 1	Lead group 2
Standard	I, II, III, aVR, aVL, aVF	V1, V2, V3, V4, V5, V6
Cabrera	aVL, I, -aVR, II, aVF, III	V1, V2, V3, V4, V5, V6

The lead group is selected in the ECG settings ([see page 52](#)).

Select speed

→ To change the printout speed (5, 12.5, **25** and 50 mm/s), press the key **"- mm/s +"**.

Select sensitivity

→ To change the printout sensitivity (5, **10** and 20 mm/mV), press the key **"- mm/mV +"**.

Stopping the printout

→ To stop the manual recording (printout), press the **Stop** key.

The printout provides the following information:


- Selected leads
- Heart rate, averaged over four beats
- Patient ID or Visit ID
- Time and date
- Speed
- Sensitivity
- Filter
- Device ID
- Serial number
- Software version

5.4 Rhythm recording

Press **Rhythm** to perform a rhythm recording. Select the recording duration in the dialogue that pops up. If a recording is canceled after more than 10 seconds, it can still be stored. The recording can be checked and saved and further printouts can be obtained in different formats. Depending on the setting, the recording is deleted automatically as soon as it has been transmitted, or it remains stored in the memory.

Rhythm

The dialogue "Rhythm settings" can be deactivated in the Settings menu or directly in the dialogue. To reactivate the dialogue, tick the option in the Rhythm ECG menu, see chapter 8.2.6.

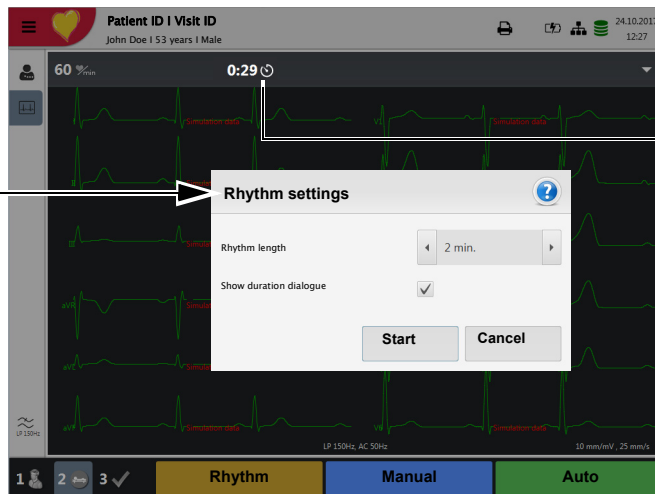


Once Start is pressed, the remaining recording duration is displayed

Cancel

After 10 s of recording, the icon Cancel changes to Stop. Press Stop to end and store the recording.

Stop

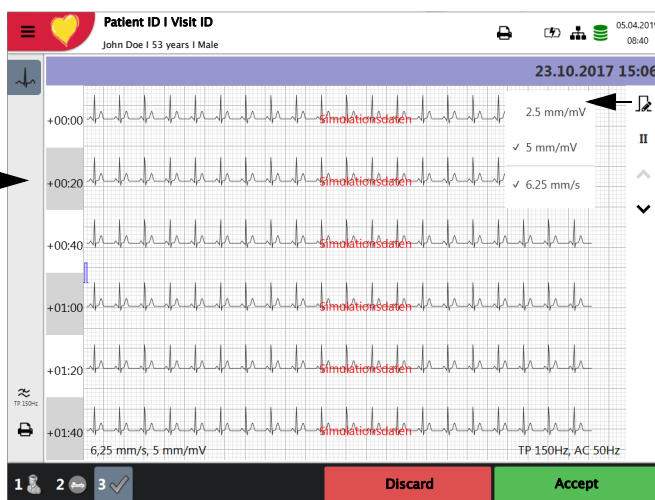

Time intervals

Set the amplitude and select the channel, I..V6

For recordings with a duration of 3 or 4 minutes, the time interval is displayed

Check the recording using **II** (select channel) and **^** (time axis).

- Select filter (Off, 25, 40, 150 Hz)
- **Accept** the recording
- Print the recording
- Press **Discard** to exit the preview without storing the ECG
- Select a recording for review and printout via the **Memory** menu
- Open the menu **Memory** to delete a recording from memory.

5.4.1 Automatic printout

The printout gives the following:

- Heart rate
- Patient ID or Visit ID
- Time and date
- Speed
- Sensitivity
- Filter
- Device ID
- Serial number
- Software version
- Resting rhythm curves, see [page 61](#).

5.5 Changing the ECG preview layout



The ECG preview is optimized for 2 columns with 6 leads each, or for 3 columns with 4 leads each. The amplitude and speed can be set to 5, **10** or 20 mm/mV, and to 12.5, **25** or 50 mm/s. For the settings, see [section 5.6.3 Views & layouts, page 54](#).

The hookup preview layout is always set to 2 columns with 6 leads each.

5.5.1 Display

Leads → The following presentation can be selected in **Menu > Settings > ECG > Leads & cable**:

The Standard and Cabrera lead sequences are as follows:


Lead sequence	Lead group 1	Lead group 2
Standard	I, II, III, aVR, aVL, aVF	V1, V2, V3, V4, V5, V6
Cabrera	aVL, I, -aVR, II, aVF, III	V1, V2, V3, V4, V5, V6

The factory setting for the Default lead configuration is Standard 12 lead. The following settings can be made:

- Standard 12-Lead
- Pediatric
- Right precordials
- Standard C4r
- Left posterior
- Nebh (chest)

5.5.2 Myogram filter (Low pass filter)

The myogram (low pass) filter suppresses disturbances caused by strong muscle tremor. In **Menu > Settings > ECG > Filters & formulas**, the **myogram filter** is defined.

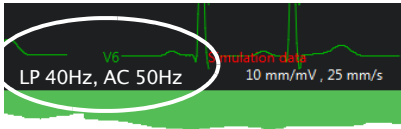

In the information field, **LP 25 Hz**, **LP 40 Hz** or **LP 150 Hz** is displayed.



- The **default** cut-off frequency is user-defined at LP 25 Hz, 40 Hz, **150 Hz** or 250 Hz (Filter Off) (see [chapter 5.5.2, page 53](#)).
- An ECG recorded in auto mode is stored unfiltered. It is therefore possible to print the stored ECG either with or without applying the myogram filter.
- For pediatric patients, we recommend setting the myogram filter to Off = 250 Hz.



- ▲ When using the 25 Hz or 40 Hz filter, the displayed or printed ECG may not always fulfill the requirements for a diagnostic ECG.

5.5.3 Other filters

The following additional filters are available:

Baseline filter

The cut-off frequency for the baseline filter is based on IEC 60601-2-25 and cannot be changed.



Notch filter



This filter prevents recording interference due to AC power frequency oscillation. If the filter is active, "AC 50 Hz" or "AC 60 Hz" is displayed.



- The filters are activated/deactivated or changed in the ECG settings (see [following description](#)).

5.6 ECG settings

When pressing the Menu key  , the option **Settings**  is displayed. The following table gives an overview of all the settings concerning the ECG acquisition:

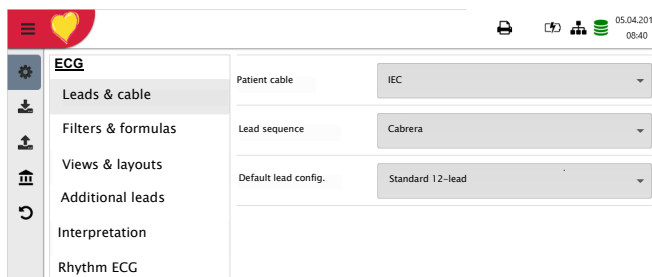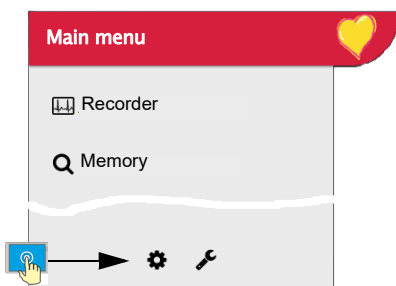

Overview **Menu > Settings**

Menu Settings	Sub-menu
ECG (Page 52)	<ul style="list-style-type: none"> Leads & cable Filters & formulas Views & layouts Additional leads Interpretation Rhythm ECG



Changed settings are saved automatically. In **Menu > Import**, settings from another device can be imported, or a backup of the settings can be restored (see [page 71](#)).

5.6.1 Leads & cable


Menu	Parameter	Description / selection
Leads & cable	Patient cable	IEC or AHA
	Lead sequence	Standard or Cabrera <ul style="list-style-type: none"> Standard 12-lead Pediatric Right precordials
	Default lead config.	<ul style="list-style-type: none"> Standard C4r Left posterior Nebh (chest) Balanced

5.6.2 Filters & formulas

Menu	Parameter	Description / selection
	Notch filter	Off/ AC 50/AC 60 Hz
Filters & formulas	Myogram filter (Low pass)	Off/LP25/LP40/LP150 Hz
	Default QTc calculation	Bazett, Fridericia, Framingham, Hodges

5.6.3 Views & layouts

In this menu, the views and layouts for the **Preview**, **RECG review** and **Resting rhythms** are defined.

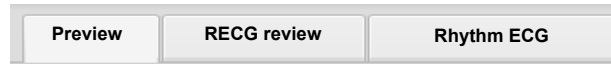


Preview

Menu	Parameter	Description
Preview	Preview view order	Select whether Hookup or Recorder is shown at the top
	9 lead layout	6/3 for Nebh Chest
	12 lead layout	2x6 / 4x3
	Speed/Amplitude	Speed 12.5/ 25 /50 mm/s Amplitude 5/ 10 /20 mm/mV

Order of preview views in Step 2




View in Step 2: either Hookup or Recorder is shown at the top

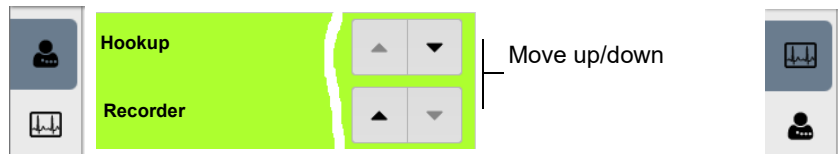


RECG review

These settings apply to current resting ECG recordings and recordings from the memory. Therefore, saved ECGs can be displayed with different settings at any time.

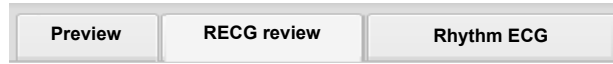


Menu	Parameter	Description
RECG Review	Review views and order	Adding/removing views, and defining their order: rhythms, averages, results and measurements. Settings see below.
	Rhythm 12 lead layout	1x6 / 1x12
	Rhythm Speed/Amplitude	Speed 12.5/ 25 /50 mm/s Amplitude 5/ 10 /20 mm/mV
	Avg. Speed/Amplitude	Speed 25 /50 mm/s Amplitude 10 /20 mm/mV

View and order in Step 3 of the Review

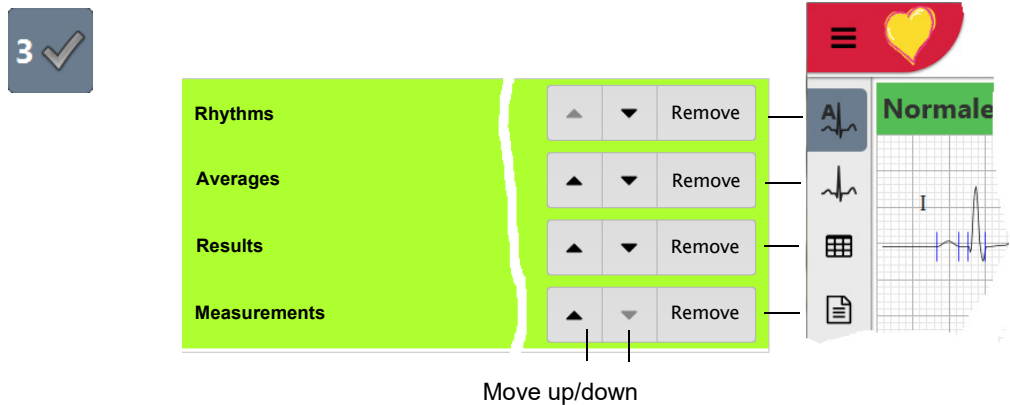


Rhythm ECG

These settings apply to current resting rhythm recordings and recordings from the memory. Therefore, saved ECGs can be displayed with different settings at any time.

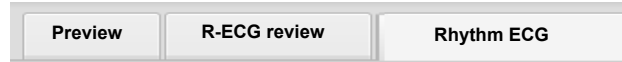

Menu	Parameter	Description
Rhythm ECG	Review views and order	Adding/removing views, and defining their order: Continuous/Rhythm and Rhythm summary. Settings see below.
	Amplitude	2.5/5 mm/mV
	Speed	Speed 6.25/12.5mm/s

View and order in Step 3 of the Review

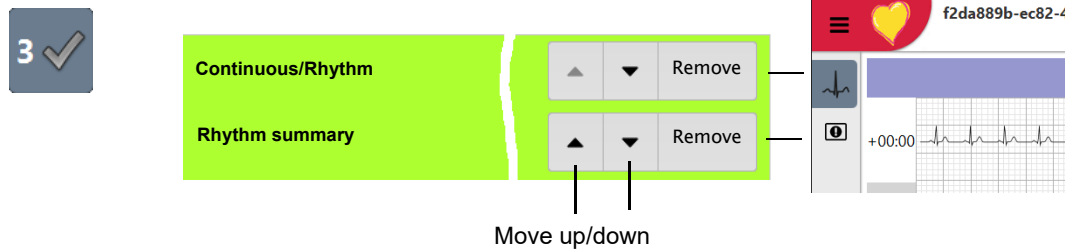

5.6.4 Additional leads

Default leads per lead configuration

These settings apply to the current resting rhythm recording, recordings from the memory, and the printout. Therefore, saved ECGs can be displayed or printed with different settings at any time.


Menu	Parameter	Description / selection
Additional leads	Standard 12-lead	I / II / III aVR / aVL / aVF / V1 / V2 / V3 / V4 / V5 / V6 / -aVR Rhythm 1 II , Rhythm 2 V2 , Rhythm 3 V5
	Paediatric	I / II / III aVR / aVL / aVF / V7 / V2 / V3r / V4r / V5 / V6 / -aVR Rhythm 1 V7 , Rhythm 2 V4r , Rhythm 3 II
	Right precordials	I / II / III aVR / aVL / aVF / V1 / V2 / V3r / V4r / V5r / V6r / -aVR Rhythm 1 V3r , Rhythm 2 V5r , Rhythm 3 II
	Standard C4r	I / II / III aVR / aVL / aVF / V1 / V2 / V3 / V4r / V5 / V6 / -aVR Rhythm 1 V4r , Rhythm 2 V2 , Rhythm 3 II
	Left posterior	I / II / III aVR / aVL / aVF / V4 / V5 / V6 / V7 / V8 / V9 / -aVR Rhythm 1 V8 , Rhythm 2 V5 , Rhythm 3 II
	Nehb (chest)	I / II / III / aVR / aVL / aVF / D / A / J / -aVR Rhythm 1 D , Rhythm 2 A , Rhythm 3 J
	Balanced	I / II / III aVR / aVL / aVF / V4r / V3r / V1 / V7 / V8 / V9 / -aVR Rhythm 1 V7 , Rhythm 2 V4r , Rhythm 3 II

5.6.5 Interpretation

Menu	Parameter	Description / selection
Interpretation	Print interpretation	Yes/No
	Display interpretation	Yes/No

5.6.6 Resting rhythm

Parameter	Description / selection
Rhythm length	30 s , 1, 2, 3 or 4 minutes Setting the recording duration.
Show recording time dialogue	<input checked="" type="checkbox"/> Yes. The dialogue can be deactivated during the recording and can be activated here again.
Predefined event texts	Enter (delete) predefined event texts which can be selected during the recording. → Press +, enter text and confirm with Enter

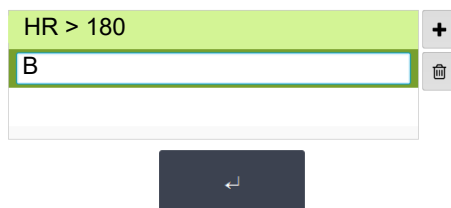

5.6.7 Reports

In this menu, the **Print layout** is defined.

General

Parameter	Description
Rhythm mode	Sequential or Simultaneous. When Sequential is selected, the individual lead groups show consecutive time segments (applies to the printout). When Simultaneous is selected, all lead groups show the same time segment (applies to the printout). If a format including a rhythm lead is defined for printout, Sequential is always used, even if you have selected Simultaneous.
PDF paper format	A4 or Letter
PDF conformance	None , PDF/A-1a, PDF/A-1b
Company info 1, 2, 3	Enter company information on PDF rows 1, 2 and 3.

Manual printout

In this menu, the default settings for manual printouts are defined.

Parameter	Description
Lead group	Select the lead group Limb or Precordials
Speed [mm/s]	5, 12.5, 25 or 50 mm/s
Amplitude [mm/mV]	5, 10 , 50 mm/mV

Resting ECG

These settings apply to current resting ECG recordings and recordings from the memory. Therefore, saved ECGs can be printed with different settings at any time.



- The order listed below can vary.

Menu	Description / selection
	<p>Selection and order of the following printout formats:</p> <ul style="list-style-type: none"> Patient data Rhythm 2x6, 25 mm/s, 1 page Averages 4x3+1, 50/25 mm/s, 1 page Results
12 Lead	<ul style="list-style-type: none"> Rhythm 2x6, 25 mm/s, 2 page Rhythm 2x6, 50 mm/s, 2 page Rhythm 4x3+1, 25 mm/s, 1 page Averages 4x3+1, 25/25 mm/s, 1 page Averages 6x2+2, 50/25 mm/s, 1 page Measurement Table
12 Lead (PDF)	<ul style="list-style-type: none"> Rhythms 10s, 25 mm/s, 2p Measurements Averages Grid, 25/25 mm/s Averages Grid, 50/25 mm/s Averages Wide, 25/25 mm/s Panorama Rhythms 10s, 25 mm/s Rhythms 5s, 25 mm/s Rhythms 5s, 50 mm/s, 2p Rhythms Grid, 25 mm/s
9 Lead	<ul style="list-style-type: none"> Patient data Rhythm 6/3, 25 mm/s, 1 page Averages 3x3+1, 50/25 mm/s, 1 page Results Rhythm 6/3, 50 mm/s, 2 pages Rhythm 6/3, 50 mm/s, 2 pages Averages 6/3+2, 50/25 mm/s, 1 page Measurement Table
9 Lead (PDF)	<ul style="list-style-type: none"> Rhythms 10s, 25 mm/s, 2p Measurements Averages Grid, 50/25 mm/s Averages Wide, 50/25 mm/s Rhythms 5s, 25 mm/s Rhythms 5s, 50 mm/s, 2p

Printout formats highlighted in green are printed.

The screenshot shows a settings menu for ECG printouts. At the top, there are four tabs: '12 lead', '12 lead (PDF)', '9 lead', and '9 lead (PDF)'. Below the tabs is a scrollable list of printout formats. The first four items are highlighted in green: 'Patient data', 'Rhythm 2x6, 25 mm/s, 1 page', 'Averages 4x3+1, 50/25 mm/s, 1 page', and 'Results'. Each of these green items has a 'Remove' button to its right. Below the green items are two more items: 'Rhythm 2x6, 25 mm/s, 2 page' and 'Rhythm 2x6, 50 mm/s, 2 page', each with an 'Append' button to its right. On the right side of the list, there are vertical arrow keys (up and down) for reordering items. A vertical scrollbar is also visible on the right edge of the list area.



Change the order by moving the items up/down using the arrow keys.

Add/remove print formats by pressing Append or Remove.

Rhythm ECG

These settings apply to current resting rhythm recordings, resting rhythm recordings from the memory (Internal, display and printout), as well as PDF. Therefore, saved resting rhythm ECGs can be printed with different settings at any time.

Menu	Description / selection
Internal	<ul style="list-style-type: none"> Amplitude 5 or 2.5 mm/mV Patient data Continuous x1, 6.25 mm/s
	<ul style="list-style-type: none"> Event list HR trend
	<ul style="list-style-type: none"> Rhythm summary Continuous 25 mm/s, 2:00 min Rhythm 5, 10, 20 mm/mV All events 2x10s, 6 pages Rhythm 5, 10, 20 mm/mV All events 2x5s, 6 pages Rhythm 5, 10, 20 mm/mV Continuous 12.5 mm/s, 5:20 min Rhythm 5, 10, 20 mm/mV
PDF	<ul style="list-style-type: none"> Continuous 6.25 mm/s, 10:40 min Rhythm 5, 10, 20 mm/mV Manual events 10s, 1/2 page Rhythm 5, 10, 20 mm/mV Manual events 10s, 1 page Rhythm 5, 10, 20 mm/mV Rhythm 10s/page Rhythm 5, 10, 20 mm/mV Rhythm 20s/page Rhythm 5, 10, 20 mm/mV

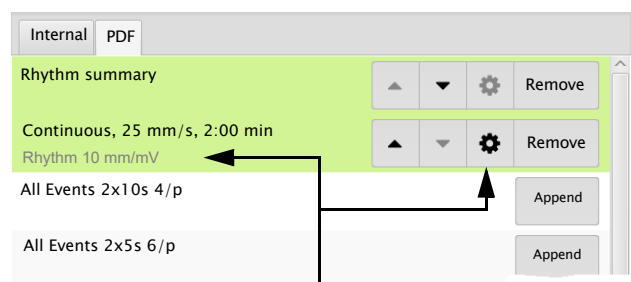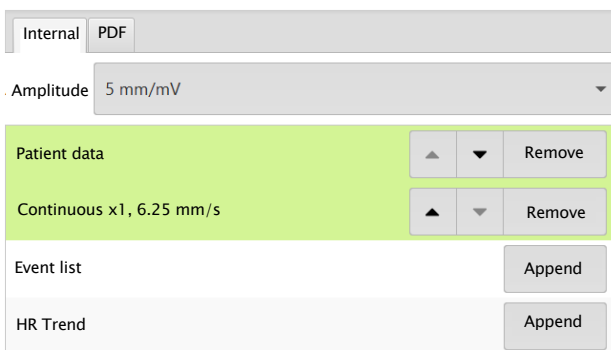

Setting the rhythm amplitude
5, 10, 20 mm/mV

6 Memory


Recordings can be stored locally and transmitted automatically to the Schiller Link or Schiller server. Recordings stored in the memory can be viewed, printed, transmitted or deleted at any time.

Approx. 350 resting ECGs and 100 resting rhythms can be stored on the CardioPulse Go. Recordings are stored manually after completion of the acquisition.



Memory capacity is indicated by the icon . A green icon means Memory OK, a yellow icon means Memory almost full, a red icon means Memory full, no recording possible. Chapter 6.1.2 describes how to delete recordings.

6.1 Editing the memory

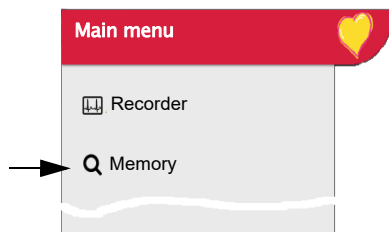

- When **Menu > Memory** is selected, stored recordings are displayed
- The recordings are listed by date/time; however, different listing criteria can be selected and recordings can also be searched via the search function.

Double click to open the recording

Sort recordings

Search recording

Selecting , transmitting or deleting one or several recordings.

Transmitting the selected recordings

Deleting the selected recordings


Recording ID	Date / time
John Doe 123456 Fall. Nr	25.10.2017 16:19
Name Surname Patient ID Cas nr.	25.10.2017 16:14
Emergency 20171025161306 4ab354c5-414e-407e-af77-737271ae6ab1	25.10.2017 16:13
Emergency 20171025160953 a80d5d92-d315-4db2-9457-5c7d366d006e	25.10.2017 16:09

Display selected recordings


View recording


6.1.1 Opening the print preview from the memory and printing a recording

Depending on the settings in **Menu > Settings > General > Workflow**, the recording is printed automatically as soon as it has been saved.


If the recording cannot be printed (e.g. no paper), the printout can be generated again by pressing the icon (1)  Print jobs in the status bar at the top.

Recordings can be searched using the Patient ID, Visit ID or first/last name. Enter the search term via keyboard or via Barcode reader.

1. Search and select the desired recording.
2. Press the key View recording.
 - The recording is displayed according to the settings in **Menu > Settings > Resting ECG > Views & layouts > RECG Review**, and the layout can be changed for the displayed recording at any time.
3. Press (2)  to print the recording in the selected format, see [section 8.2.7 Reports, page 75](#).

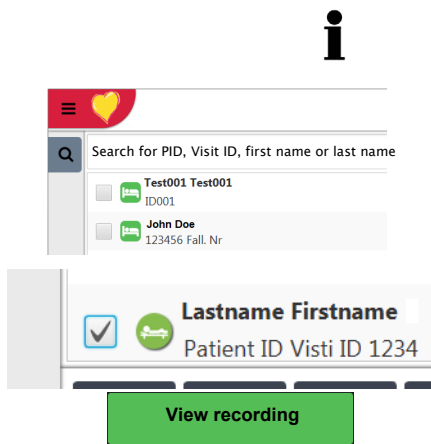


Example: resting ECG

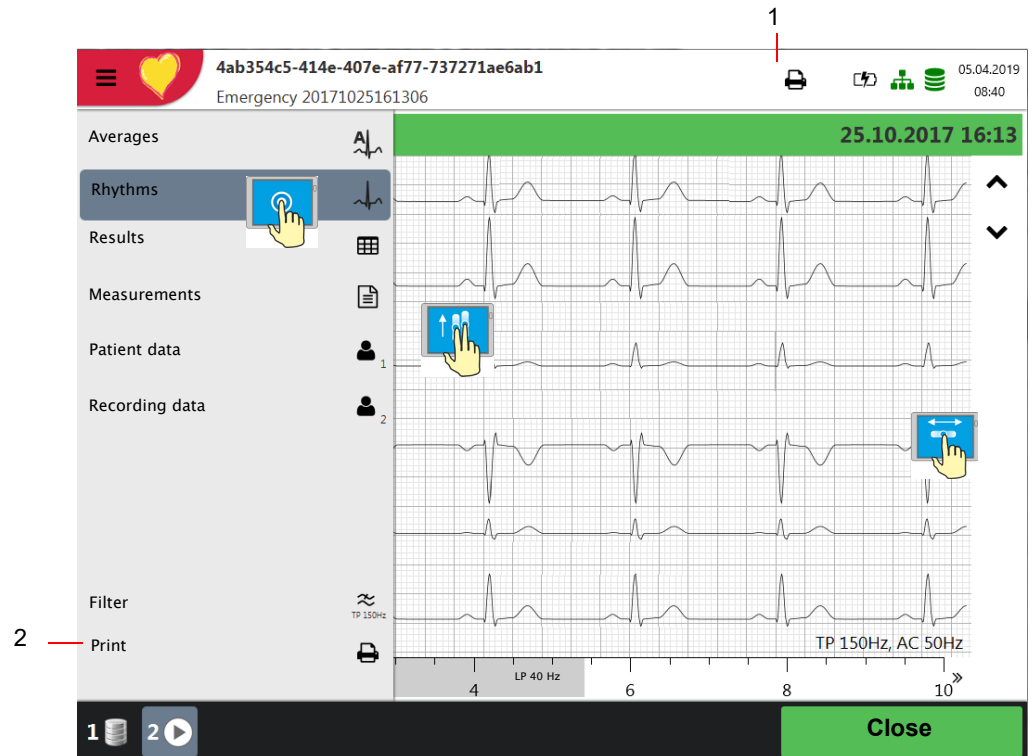

6.1.2 Transmitting and deleting stored recordings

Depending on the settings in **Menu > Settings > General > Workflow**, the recording is transmitted and deleted automatically as soon as it has been recorded. If automatic transmission is not activated, recordings can be transmitted as follows.

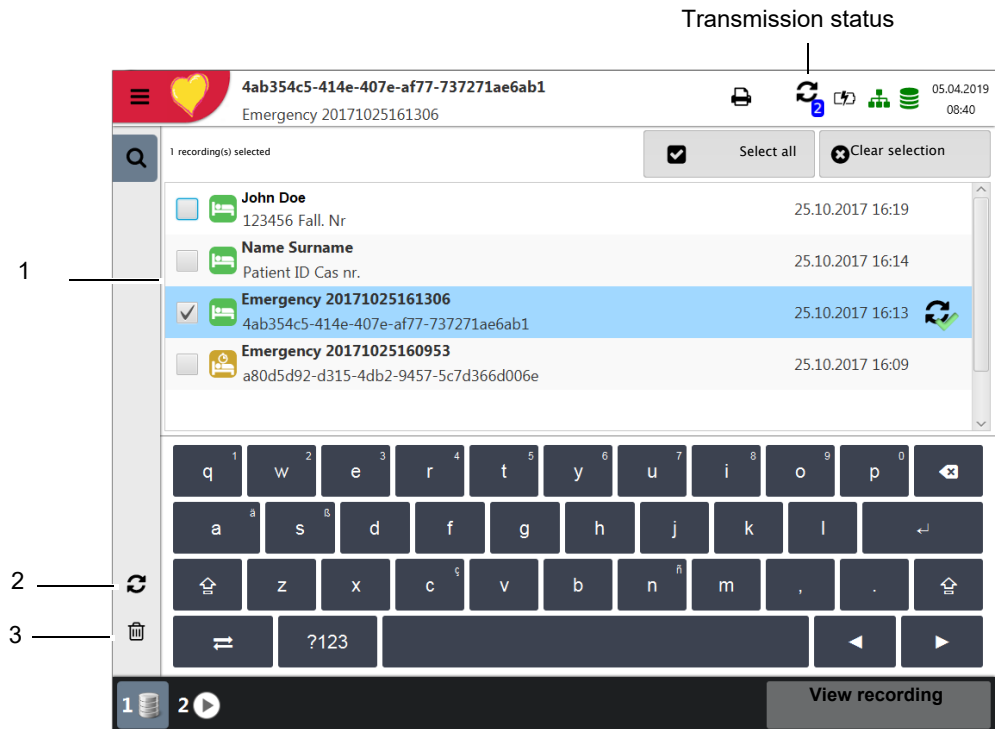

1. Select one or more recordings directly on the screen (1).
2. To select all recordings, press the key **Select all**.
3. Select one of the following options:
 - Export (2)
 - Delete (3) (automatic deletion after transmission can be set in **Settings > General > Workflow**).

If the selected operation cannot be carried out due to an error, an error message is issued (see [page 87](#)) and the following symbols/buttons are displayed for each recording:



Indicates faulty recordings or recordings that have not been transmitted.



Indicates recordings that have not been transmitted.



Exported recordings.



The transmitting options are detailed in the section System settings (see [page 83](#)).

7 Worklist (Option)

7.1 General information

The Worklist function enables a doctor / administrator to define a worklist of patients that require recordings to be made. The doctor can define the patient, room / department, and specify the type of recording to be made. The worklist is defined directly from the Hospital information system (HIS) (e.g. Server or GDT) and after the recording has been made by the CardioPulse Go, the recordings are sent back to the HIS for analysis, examination and storage.

Instead of the type of recording, "Undefined" can be set. When this is the case, only the patient demographics are sent to the unit.




- To be able to use the worklist function, the unit must be set up to communicate with Server (see [page 78](#)).
- The Server user guides describes how to define worklists from Server.



The server has options to send a worklist to a specific unit or to all units on the system. To receive a specific worklist from server, the unit identification of the CardioPulse Go (device ID in the system) must be the same as the one defined for the server. This is usually set when the unit is first commissioned. The Device ID is shown in **Menu > Settings > General > Station**.

7.1.1 Worklist settings


If worklists are to be used, the workflow can be adapted accordingly. To do so, set the Default workflow in **Menu > Settings > General > Workflow** to Worklist. In this way, the worklist is shown directly after power-up. However, worklist can also be selected manually from the menu.

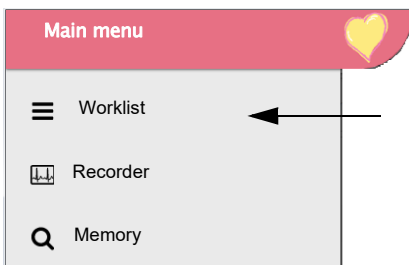

7.2 Receiving a worklist

To open the worklist proceed as follows:

1. Press **Menu > Worklist**.

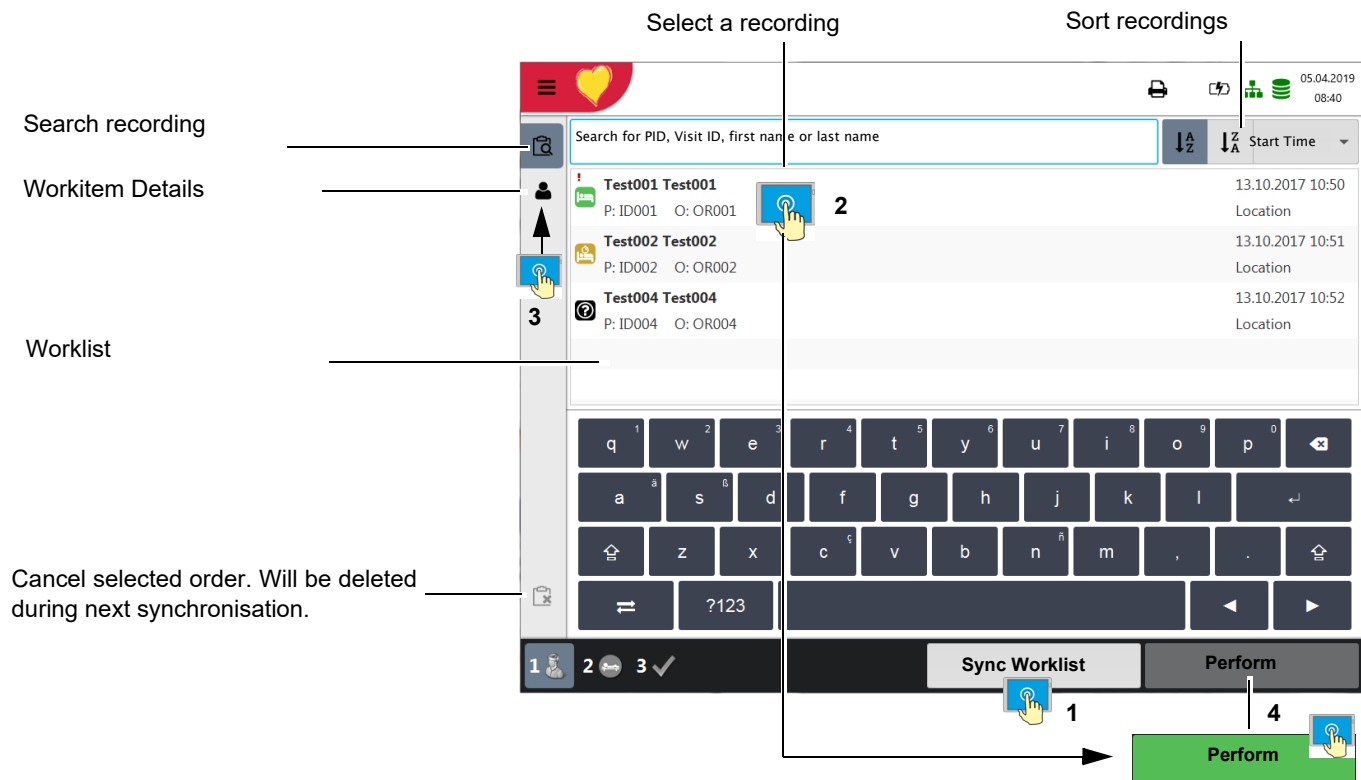

Cancel selected order. Will be deleted during next synchronisation.



If **Menu > Settings > General > Workflow** Auto sync worklist is activated, the list is updated every minute and the time for the next update is shown on the key.

2. To receive a worklist from a HIS, press the **Sync worklist (1)** key to download the worklist from the server. Wait (up to a few minutes) for the worklist to be populated. Select a work item **(2)**. You can check the work item by pressing **(3)**, return to the worklist and/or perform the work item **(4)**.
3. All patients given in the worklist are displayed with their last/first name, patient ID, order ID and room number. The following recording types are available:

- Resting ECG
- Resting rhythm
- Undefined recording type. The recording type is assigned when the recording is performed.



Recording status:


- White background = recording to be performed.
- Blue or grey background = selected recording.
- Green background = already performed recording. This work item will be deleted from the device during the next synchronization.
- Red background = recording was canceled. Will be deleted during next synchronization.

	Test001 Test001 P: ID001 O: OR001	13.10.2017 10:50 Location
	Test002 Test002 P: ID002 O: OR001	13.10.2018 10:51 Location
	Test004 Test004 P: ID004 O: OR004	13.10.2017 10:52 Location
	Test005 Test005 P: ID001 O: OR001	13.10.2017 10:50 Location


7.2.1 Taking a Worklist Recording



- This procedure corresponds with the worklist mode “Record from worklist”, see setting [section 8.5 General, page 80, Workflow](#).
- Patient data provided by the HIS cannot be edited (except for height and weight).
- If you have selected an incorrect work item, use the key  (and **not**)  to cancel the selected work item. Select the new work item from the list, or use the Search field.

1. Prepare the patient and select a work item.
2. Select **Work item details**  to check the work order or to complement patient data.
3. Press the Perform key.
4. The corresponding recording acquisition screen (resting ECG or resting rhythm) is opened. If no recording type has been defined, both options are available.





Press  to return to the worklist without performing the recording (last chance to do so).

5. Take the recording:
 - Resting ECG (see [page 45](#))
 - Resting rhythm (see [page 48](#))

7.2.2 Performing a recording from work order details





- This procedure corresponds with the worklist mode “Record from work order details”, see setting [section 8.5 General, page 80, Workflow](#).
- Patient data provided by the HIS cannot be edited (except for height and weight).
- If you have selected an incorrect work item, use the key  (and **not**)  to cancel the selected work item. Select the new work item from the list, or use the Search field.


1. Prepare the patient and select a work item.

Select a recording


Workitem Details

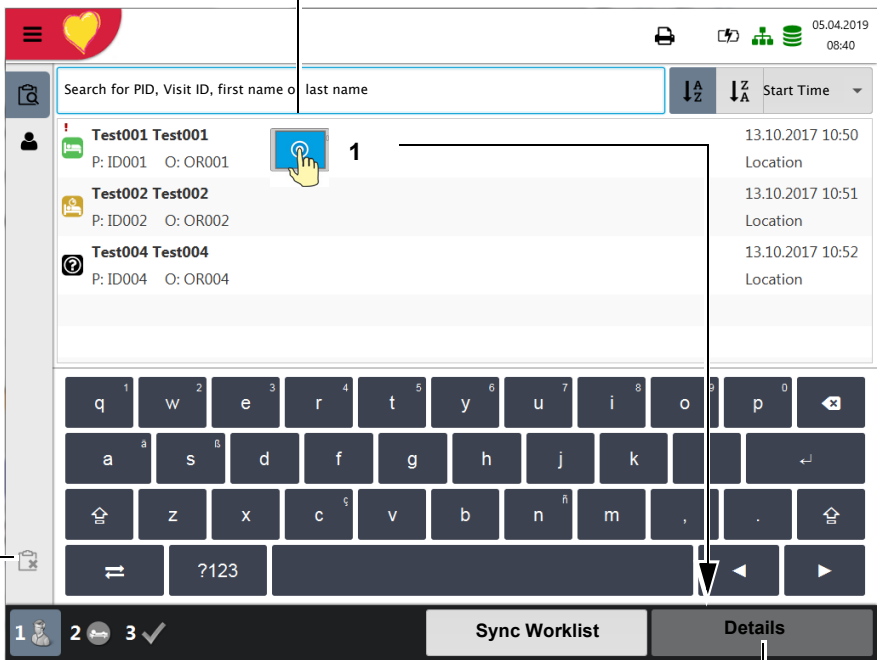



2. Select **Details (2)** to check the work order or to complement patient data.
3. Press **Perform (3)** in the “Work item details” view .
4. The corresponding recording acquisition screen (resting ECG or resting rhythm) is opened. If no recording type has been defined, both options are available.



Press  to return to the worklist without performing the recording (last chance to do so).

5. Take the recording:
 - Resting ECG (see [page 45](#))
 - Resting rhythm (see [page 48](#))

7.2.3 Sending worklist recordings to the HIS



- It is possible to automatically send performed worklist recordings; alternatively, you can have the unit prompt you to manually send worklist items. This is defined in system settings (**Menu > Settings > General > Workflow** [page 80](#)).



- In order to manually transmit recordings to server, press **Sync. worklist**. Wait (up to a few minutes) while the recordings are sent - the Worklist will go blank after the files have been sent.



Pending work items are indicated by a white background and selected work items by a blue or grey background. Already performed recordings (green background) can be sent at any time without losing pending work items.



	Test001 Test001 P: ID001 O: OR001	13.10.2017 10:50 Location
	Test002 Test002 P: ID002 O: OR001	13.10.2018 10:51 Location
	Test004 Test004 P: ID004 O: OR004	13.10.2017 10:52 Location
	Test005 Test005 P: ID001 O: OR001	13.10.2017 10:50 Location

Canceled work items (red background) are deleted from the worklist during the next synchronization.

8 General and system settings

8.1 System settings

When pressing the Menu key, the options Settings and Memory are displayed. The following table gives an overview of all the settings available.

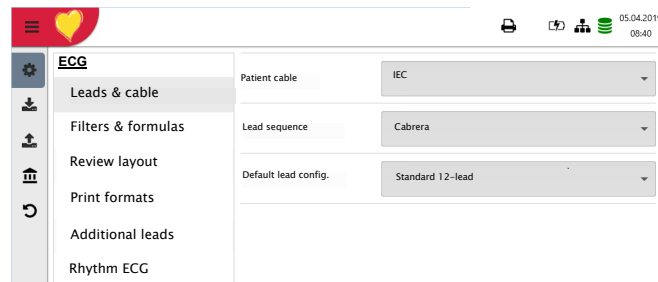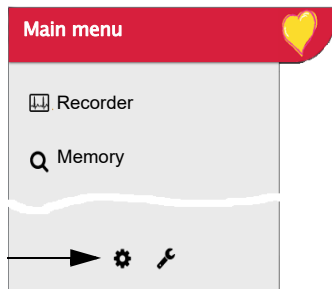

Overview **Menu > Settings**

Menu Settings	Sub-menu
ECG (Page 72)	<ul style="list-style-type: none"> Leads & cable Filters & formulas Views & layouts Additional leads Interpretation Rhythm ECG
Reports (Page 75)	<ul style="list-style-type: none"> General Manual printout Resting ECG Rhythm ECG
Connectivity (Page 78)	<ul style="list-style-type: none"> EMR integration Update server Ethernet WLAN
Regional (Page 79)	<ul style="list-style-type: none"> Date & time Keyboard Language Units Patient ID system

Menu Settings	Sub-menu
General (Page 80)	• Info
	• Power management
	• Station
	• Update
	• Manage licenses
	• Mandatory fields
	• Workflow
	• Access control
	• Printer

8.1.1 Saving and restoring settings



The settings are saved automatically.

Import settings 

Select Import and Import target USB and enter the desired file name to load the stored settings (e.g. default settings) from the USB to the device.

Export settings 

Select Export and Export target USB and enter a file name to save the settings.

Reset to factory settings 

All settings are reset to the factory defaults. If the network settings are to be reset as well, untick the check box.

8.2 ECG

The default settings are printed **bold**.

8.2.1 Leads & cable

Menu	Parameter	Description / selection
Leads & cable	Patient cable	IEC or AHA
	Lead sequence	Standard or Cabrera <ul style="list-style-type: none"> • Standard 12-lead • Pediatric • Right precordials
	Default lead config.	<ul style="list-style-type: none"> • Standard C4r • Left posterior • Nebh (chest) • Balanced


8.2.2 Filters & formulas

Menu	Parameter	Description / selection
Filters & formulas	Notch filter	Off/AC 50/AC 60 Hz
	Myogram (Low pass) filter	Off = 250 Hz/LP25/LP40/LP150 Hz
	Default QTc calculation	Bazett, Fridericia, Framingham, Hodges

8.2.3 Views & layouts


In this menu, the views and layouts for the **Preview**, **RECG review** and **Resting rhythms** are defined.

Preview

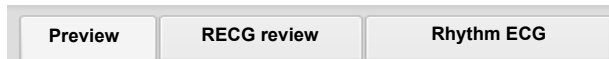

Menu	Parameter	Description
Preview	Preview view order	Select whether Hookup or Recorder is shown at the top
	9 lead layout	6/3 for Nebh Chest
	12 lead layout	2x6 / 4x3
	Speed/Amplitude	Speed 12.5/ 25 /50 mm/s Amplitude 5/ 10 /20 mm/mV

RECG Review

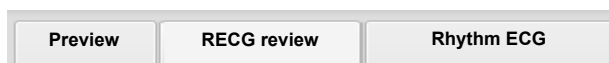

These settings apply to current resting ECG recordings and recordings from the memory. Therefore, saved ECGs can be displayed with different settings at any time.

Menu	Parameter	Description
RECG Review	Review views and order	Adding/removing views, and defining their order: rhythms, averages, results and measurements.
	Rhythm 12 lead layout	1x6 / 1x12
	Rhythm Speed/Amplitude	Speed 12.5/ 25 /50 mm/s Amplitude 5/ 10 /20 mm/mV
	Avg. Speed/Amplitude	Speed 25 /50 mm/s Amplitude 10 /20 mm/mV

Rhythm ECG

These settings apply to current resting rhythm recordings and recordings from the memory. Therefore, saved ECGs can be displayed with different settings at any time.



Menu	Parameter	Description
Rhythm ECG	Review views and order	Adding/removing views, and defining their order: Continuous/Rhythm and Rhythm summary. Settings see below.
	Amplitude	2.5/ 5 mm/mV
	Speed	Speed 6.25 /12.5mm/s

8.2.4 Additional leads

Default leads per lead configuration


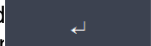
These settings apply to the current resting rhythm recording, recordings from the memory, and the printout. Therefore, saved ECGs can be displayed or printed with different settings at any time.

Menu	Parameter	Description / selection
Additional leads	Standard 12-lead	I / II / III aVR / aVL / aVF / V1 / V2 / V3 / V4 / V5 / V6 / -aVR Rhythm 1 II , Rhythm 2 V2 , Rhythm 3 V5
	Pediatric	I / II / III aVR / aVL / aVF / V7 / V2 / V3r / V4r / V5 / V6 / -aVR Rhythm 1 V7 , Rhythm 2 V4r , Rhythm 3 II
	Right precordials	I / II / III aVR / aVL / aVF / V1 / V2 / V3r / V4r / V5r / V6r / -aVR Rhythm 1 V3r , Rhythm 2 V5r , Rhythm 3 II
	Standard C4r	I / II / III aVR / aVL / aVF / V1 / V2 / V3 / V4r / V5 / V6 / -aVR Rhythm 1 V4r , Rhythm 2 V2 , Rhythm 3 II
	Left posterior	I / II / III aVR / aVL / aVF / V4 / V5 / V6 / V7 / V8 / V9 / -aVR Rhythm 1 V8 , Rhythm 2 V5 , Rhythm 3 II
	Nehb (chest)	I / II / III / aVR / aVL / aVF / D / A / J / -aVR Rhythm 1 D , Rhythm 2 A , Rhythm 3 J
	Balanced	I / II / III aVR / aVL / aVF / V4r / V3r / V1 / V7 / V8 / V9 / -aVR Rhythm 1 V7 , Rhythm 2 V4r , Rhythm 3 II

8.2.5 Interpretation

Menu	Parameter	Description / selection
Interpretation	Print interpretation	Yes/No
	Display interpretation	Yes/No

8.2.6 Resting rhythm

Parameter	Description / selection
Rhythm length	30 s , 1, 2, 3 or 4 minutes Setting the " " " " " "
Show recording time dialogue	<input checked="" type="checkbox"/> Yes. HR > 180 The dialog B and can be 
Predefined event texts	Enter (delete) predefined  which can be selected during the recording → Press +, enter text and confirm with Enter.

8.2.7 Reports

In this menu, the **Print layout** is defined.

General

Parameter	Description
Rhythm mode	Sequential or Simultaneous. When Sequential is selected, the individual lead groups show consecutive time segments (applies to the printout). When Simultaneous is selected, all lead groups show the same time segment (applies to the printout). If a format including a rhythm lead is defined for printout, Sequential is always used, even if you have selected Simultaneous.
PDF paper format	A4 or Letter
PDF conformance	None , PDF/A-1a, PDF/A-1b
Company info 1, 2, 3	Enter company information on PDF rows 1, 2 and 3.

Manual printout

In this menu, the default settings for manual printouts are defined.

Parameter	Description
Lead group	Select the lead group Limb or Precordials
Speed [mm/s]	5, 12.5, 25 or 50 mm/s
Amplitude [mm/mV]	5, 10 , 50 mm/mV

Resting ECG

These settings apply to current resting ECG recordings and recordings from the memory. Therefore, saved ECGs can be printed with different settings at any time.





- The order listed below can vary.

Menu	Description / selection
	<p>Selection and order of the following printout formats:</p> <ul style="list-style-type: none"> • Patient data • Rhythm 2x6, 25 mm/s, 1 page • Averages 4x3+1, 50/25 mm/s, 1 page • Results
12 Lead	<ul style="list-style-type: none"> • Rhythm 2x6, 25 mm/s, 2 page • Rhythm 2x6, 50 mm/s, 2 page • Rhythm 4x3+1, 25 mm/s, 1 page • Averages 4x3+1, 25/25 mm/s, 1 page • Averages 6x2+2, 50/25 mm/s, 1 page • Measurement Table
12 Lead (PDF)	<ul style="list-style-type: none"> • Rhythms 10s, 25 mm/s, 2p • Measurements • Averages Grid, 25/25 mm/s • Averages Grid, 50/25 mm/s • Averages Wide, 25/25 mm/s • Panorama • Rhythms 10s, 25 mm/s • Rhythms 5s, 25 mm/s • Rhythms 5s, 50 mm/s, 2p • Rhythms Grid, 25 mm/s
9 Lead	<ul style="list-style-type: none"> • Patient data • Rhythm 6/3, 25 mm/s, 1 page • Averages 3x3+1, 50/25 mm/s, 1 page • Results • Rhythm 6/3, 50 mm/s, 2 pages • Rhythm 6/3, 50 mm/s, 2 pages • Averages 6/3+2, 50/25 mm/s, 1 page • Measurement Table
9 Lead (PDF)	<ul style="list-style-type: none"> • Rhythms 10s, 25 mm/s, 2p • Measurements • Averages Grid, 50/25 mm/s • Averages Wide, 50/25 mm/s • Rhythms 5s, 25 mm/s • Rhythms 5s, 50 mm/s, 2p

Rhythm ECG

These settings apply to current resting rhythm recordings, resting rhythm recordings from the memory (Internal, display and printout), as well as PDF. Therefore, saved resting rhythm ECGs can be printed with different settings at any time.


Menu	Description / selection
Internal	<ul style="list-style-type: none"> Amplitude 5 or 2.5 mm/mV Patient data Continuous x1, 6.25 mm/s
	<ul style="list-style-type: none"> Event list HR trend
	<ul style="list-style-type: none"> Rhythm summary Continuous 25 mm/s, 2:00 min Rhythm 5, 10, 20 mm/mV All events 2x10s, 6 pages Rhythm 5, 10, 20 mm/mV All events 2x5s, 6 pages Rhythm 5, 10, 20 mm/mV Continuous 12.5 mm/s, 5:20 min Rhythm 5, 10, 20 mm/mV
PDF	<ul style="list-style-type: none"> Continuous 6.25 mm/s, 10:40 min Rhythm 5, 10, 20 mm/mV Manual events 10s, 1/2 page Rhythm 5, 10, 20 mm/mV Manual events 10s, 1 page Rhythm 5, 10, 20 mm/mV Rhythm 10s/page Rhythm 5, 10, 20 mm/mV Rhythm 20s/page Rhythm 5, 10, 20 mm/mV

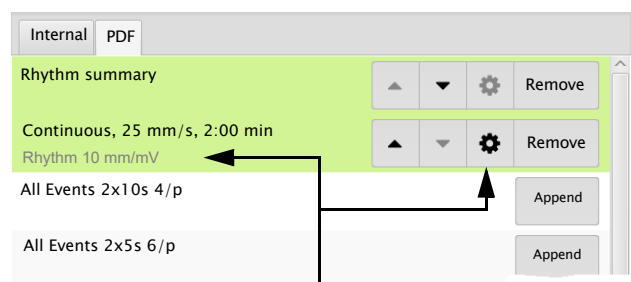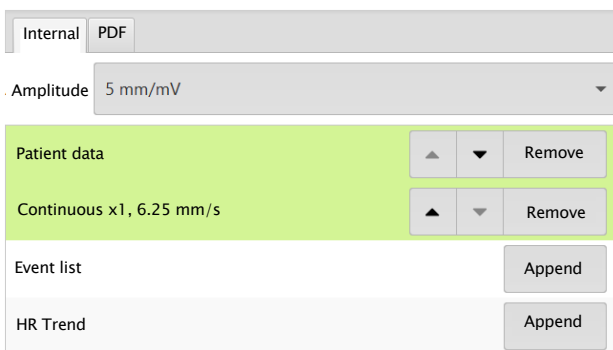

Setting the rhythm amplitude
5, 10, 20 mm/mV

8.3 Connectivity


Menu	Parameter	Description / selection
EMR integration Server settings	EMR integration (EMR = electronic medical record system)	None – No input field displayed Schiller Link – Device ID is displayed Schiller Server – Host, port, user and password input fields are displayed. (See following)
	Host	Name of the server
	SSL cert. validation	No/Yes <input checked="" type="checkbox"/>
	Port	Port address
	User	User name
	Password	Password
Update server	Host	Schiller Update server
	Port	Port address and key Test connection
Ethernet	Use DHCP	<input checked="" type="checkbox"/> Active or <input type="checkbox"/> Not active. If this is not activated, the following parameters need to be entered:
	IP address	Identifier address of the device in the TCP/IP network.
	Subnet mask	E.g.: 255.255.255.0
	Standard Gateway	Gateway IP address.
	DNS server	Domain name of the server
WLAN general	Wi-Fi enabled	<input checked="" type="checkbox"/> Active <input type="checkbox"/> Not active
	SSID..... <input type="checkbox"/> Hidden	SSID = Enter network name. Check box "Hidden" = if the SSID is configured as hidden in the Wi-Fi network, the check box "SSID Hidden" needs to be activated.
	Wi-Fi security	Selection of the encryption protocol <ul style="list-style-type: none"> • WPA/WPA2 personal SSID + key + (encryption = AES + authentication) • WPA2 enterprise / ieee802.1 (<i>additional settings see *</i>) SSID + certificate + (encryption = AES + authentication) SSID + user name & password + (encryption = AES + authentication)
WLAN security (for WPA/WPA2 personal)	Password	Enter password for WPA/WPA2 protocol



Menu	Parameter	Description / selection
WLAN security (for WPA2 enterprise / ieee802.1)	Authentication protocol	Select the authentication protocol: PEAP , EAP-TLS or EPA-TTLS
	User	Definition of the user name
	Password	Definition of the password
	Client certificate and CA certificate	<p>Download the certificate via USB port of the device when EAP-TLS is selected</p> <p>→ Connect USB stick to the device and press Load  .</p> <p>Certificate structure: a single file in pem. format contains client certificate, of origin, private key. The private key may or may not be encrypted. If it is encrypted, the user name and password need to be entered.</p>
Advanced	<input type="checkbox"/> Hidden	Check box "Hidden" = if the SSID is configured as hidden in the Wi-Fi network, the check box "SSID Hidden" needs to be activated.
	Anonymous identity	Enter an anonymous identity (name)
WLAN network	see Ethernet parameters	Settings network parameters

8.4 Regional settings

Sub-menu	Parameter	Description / selection
Date/time	Various	Date and time format Time zone Date and time setting → Key Sync time with server. Time and date on the device are updated. The device needs to be restarted.
		Keyboard layout
Keyboard	External keyboard layout (barcode scanner layout)	Selecting the character set for the external barcode scanner.
Language	Language	Select a language
Units	Weight	Units available are g, kg and lb
	Length	cm , m, inch
	Temperature	Celsius or Fahrenheit
Patient ID system	None	Using country-specific patient IDs.
	Swedish	
	Danish	
	Finnish	
	Norwegian	

8.5 General

Menu	Parameter	Description / selection
	Various parameters	Software and hardware versions are displayed.
Info		A diagnostic file (.nfo) is written to the connected USB USB memory stick.
Power management	Battery operation Dim backlight [s]	120 seconds (2 min.). If set to 0, this function is deactivated.
	Shut down device [s]	600 seconds (5 min.). If set to 0, this function is deactivated.
	AC power operation Dim backlight [s]	0 seconds (0 min.). If set to 0, this function is deactivated.
	Shut down device [s]	3600 seconds (1 hour). If set to 0, this function is deactivated.
Station	Device ID	Device identification
	Institute	Name of the institute
	Department	Name of the department
	Technician	Technician
	Network host name	Host name of the device (e.g. ft1) displayed in the network.
Update	Check update Schiller server/ Gateway	Update software The check is performed on the update server. Therefore, an Ethernet/ WLAN connection is required, including the necessary network set- tings for this connection
	Check USB device for update file	The Update is performed via the connected USB stick.
Manage licenses	Available options	Automatic interpretation Worklist
	Activate license	Enter the license key and activate
	Import license from USB	Activation via USB stick (.lic file)
Mandatory fields	Defining the mandatory fields	Following fields must be filled before a recording can be taken. Mandatory fields for exercise ECG which cannot be deactivated: Gender, height, weight, date of birth.

Menu	Parameter	Description / selection
	Transmit after acquisition	ECG data is transmitted after acquisition and storage of the recording
	PDF to USB after save	Yes, No . After saving, the PDF is automatically exported to the USB stick
	Delete after transmission	ECG data is deleted once it has been successfully transmitted from the memory.
	Print after acquisition	ECG data is printed once it has been stored.
Workflow	PDQ mode	PDQ by Patient ID PDQ by Visit ID None (PDQ is not started, even if EPA integration has been selected)
	Default workflow	Worklist Recorder
	Worklist mode (only if option has been activated)	Record from worklist Record from work order details 
	Auto sync worklist	<input checked="" type="checkbox"/> Yes, No . Worklist is synchronized every minute.
	Access control mode	<ul style="list-style-type: none"> • Basic Login when switching on the device and/or accessing the Settings menu requires a password • Local (Default factory setting) Definition of users, passwords and privileges locally on the device • Schiller Server Access control is defined via the Schiller Server.
	Basic	
	Device login active	<input checked="" type="checkbox"/> If activated, the login dialogue is displayed at switch-on.
Access control (automatic logout when access control is activated, see menu "Automatic logout" next page)	Device password	Defining the password (default)
	Settings Login Active	<input checked="" type="checkbox"/> If this is activated, the login dialogue is displayed when the settings are opened 
	Settings Password	Defining the password (admin)
	Local	
	User name	Administrator (additional users can be created)
Password	Enter the password ("administrator" is used as password if no other password is entered)	
Retype Password	Confirm password	
Privileges	Selecting the privileges: Adjust system settings; Analyze recordings (from memory); Create recordings.	
Schiller Server	This requires a working EMR connection and Schiller Server administrator rights. Access control is defined on the Schiller Server.	


REF: 2.511461 REV B / CAT: 070-3062-00 REV B

Menu	Parameter	Description / selection
Automatic logout (Is only displayed if Access control is activated)	Automatic logout enabled	<input checked="" type="checkbox"/> Yes/No
	Logout timeout [s]	300
Print setting	Contrast	1-10 (5)
	Line width	Thin, normal , thick

8.5.1 Setting user name and password for access control



- ▲ To prevent non-authorized users to access patient information, manipulate settings or the software, we strongly recommend changing the default password to a strong password according to the standard rules.
- ▲ Do not switch off access control if the device is connected to a network.

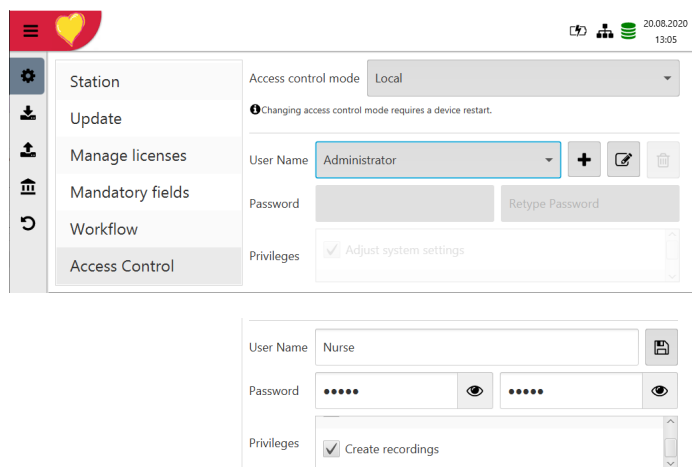

1. Choose the “Access control” menu. The Access control mode “Local” is by default activated.
2. Select one of the predefined “User Name” (Nurse, Physician) by pressing the key.
3. Use the buttons to add, edit or delete users.
 → User privileges can be selected or deselected with the check boy . The “Adjust system settings” privilege cannot be disabled for the administrator.
4. Select the button to save the settings.

9 Transmission - Overview



- ▲ Security of the network is the sole responsibility of the network operator.
- ▲ Spacelabs Healthcare takes no responsibility for the configuration of Windows.
- ▲ In order to guarantee the security of the network, Spacelabs Healthcare recommends the following:
 - isolating the CardioPulse Go network from other networks
 - defining access authorization for the configuration of the host system, incl. CardioPulse Go, so that no unauthorized alterations of the system are possible
 - limiting the data transmission between the host and other systems/networks to a minimum
 - installing the latest anti virus/firewall programs on the host in order to prevent malware from affecting the system
 - regularly installing security updates on the host
 - installing software updates that increase the CardioPulse Go's security
 - taking the appropriate measures to check the system's security and ensure safe operation when changing the network configuration, installing security updates and adding/removing devices.

9.1 Transmission Options

With the CardioPulse Go, transmission is possible via a network or Wi-Fi. The transmission options are as follows:



- ▲ When a non-medical device is connected to the interface, ensure that both units are securely connected to the same earth potential.
- ▲ An external device must only be connected using the original interface cable assembly.
- ▲ The transmission of ECG data via WLAN can disturb other devices, including pacemakers. Therefore, keep a distance of at least 20 cm from the patient while an ECG is transmitted.

LAN


data transmission via local LAN network (Ethernet) to the EMR system. For an Ethernet (network) connection, connect the cable assembly to the RJ-45 connector.

LAN or WLAN not active/active

The network symbol in the status bar at the top right indicates with a square ■ that the connection (WLAN or LAN) is active.

Green = connection to network/server established

Black = no connection to network/server

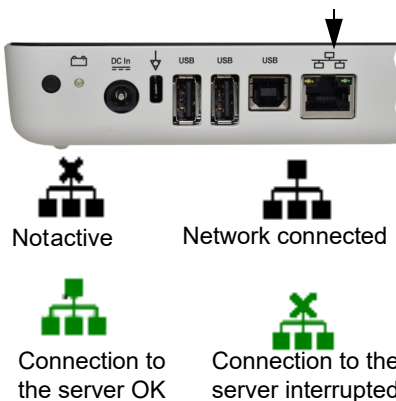

Wifi

If Wifi is activated, the following symbols are displayed



- Green symbol: connected to the Wifi network and the Server.
- Black Symbol: connected to the network but no connection to the Server
- Signal strength is displayed with bars.

Schiller Link/Schiller Gateway

Schiller Link/Schiller Gateway offers easy communication with an EMR system within the same network. This communication comprises the following: import (GDT) of examination requests including patient data and recording type from an EMR system, export of recordings to an EMR system in the formats GDT, Sema2 or PDF. To activate this communication, set **Schiller Link** in the menu Connectivity > EMR integration (see page 78).

The transmitted patient data is displayed automatically when the patient data screen is open, if it does not contain any entries and is not in review mode after a recording.

Schiller Server/Schiller Gateway

For patient data queries from the EMR system, the Schiller Server/Schiller Gateway is required.

PDF export

Simple export of a recording as PDF to a memory stick

9.1.1 Automatic transmission



The automatic transmission setting is defined in Settings:

Menu > Settings > General - Workflow- Transmit after acquisition (Yes/No - see page 80).

When auto transmission is defined, a recording is transmitted automatically after it has been saved.

9.1.2 Manual transmission

To transmit a recording, select the recording in the **Memory** and press **Export** (see page 64)

9.1.3 PDF export

Data integrity




- ▲ When exporting patient data to a memory stick, the responsible user needs to protect the data with appropriate security measures:
 - Make sure that only authorized people have access to the memory stick.
 - Once the data has been transmitted from the memory stick to a secure system, delete all data from the memory stick.
 - Deactivate the PDF export function if it is not used.

Settings




Activate **PDF export** in the **Menu > Settings > Workflow > PDF to USB after save**. If **PDF export** is active, the recordings from the memory are transmitted as soon as a USB stick is connected. The symbol PDF export is displayed when data has successfully been transferred to the memory stick.

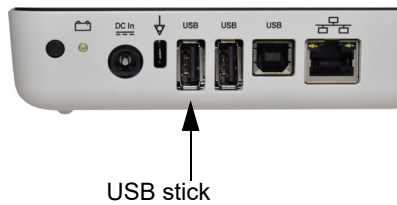

Caution

If **“Delete after transmission”** is activated in the same menu, the recordings are deleted from the memory.

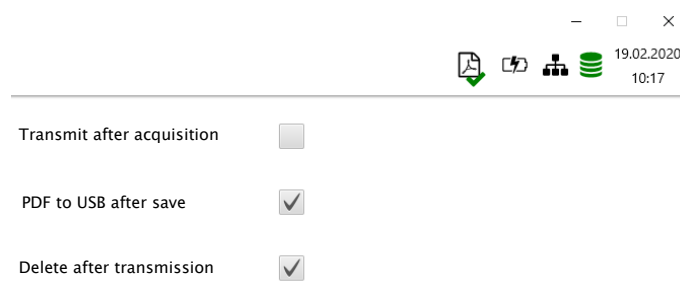

Manual PDF export from memory

An individual recording can be exported to a USB stick from the memory if the setting above **“Delete after transmission”** has been selected.




- Open the Memory, select the recording and export with the key. Storage progress is shown in the status bar with the PDF symbol.

9.1.4 Schiller Link

Schiller Link is a PC application/service which communicates between the EMR system and the CardioPulse Go.

- To activate this communication, set Schiller Link in the menu **Connectivity > EMR integration** (see page 78).
- Integration in the network is automatic, provided the CardioPulse Go is part of the same network

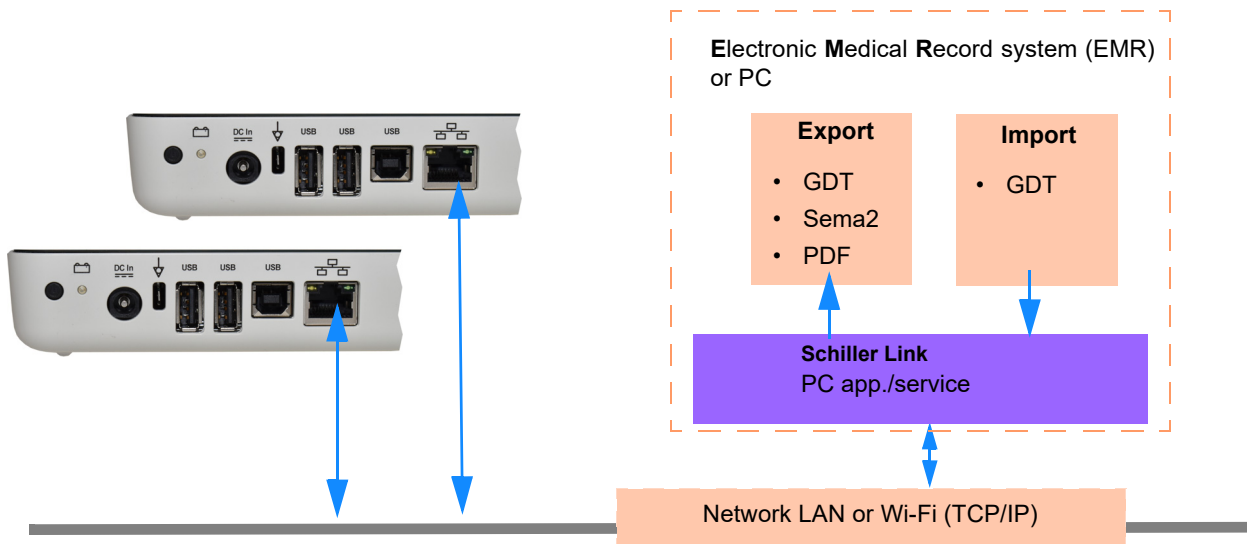

Procedure with EMR system

1. Enter/select a patient in the EMR system
2. Generate a new order for this patient
3. Upload the GDT file into the import folder of the Schiller Link service
4. Start and check the order incl. patient data on the CardioPulse Go.
5. Perform the recording on the CardioPulse Go.
6. Store the recording and export it automatically or manually to the export folder.
7. The EMR system imports the recording for review in the EMR system.

Procedure without EMR system

1. Manually enter the patient data on the CardioPulse Go (via keyboard or Barcode reader).
2. Perform the recording on the CardioPulse Go.
3. Store the recording and export it automatically or manually to the export folder.
4. Review the recording (PDF) on the PC and print or transmit it via e-mail.


9.1.5 Receiving data from server

Patient data can be received from a server and automatically entered on the CardioPulse Go. This is called patient data query (PDQ). To do this, the Patient or Visit ID is entered in the patient data screen manually or via a Barcode reader (see page 30).



- Patient data query requires the Server being available via network.
- The server name, URL, TCP/IP address etc. as well as all other transmission settings are defined in the system settings (see page 78).

9.1.6 Failed data transmission


Failed transmission of data is indicated with the symbol  in the status bar.



1. The blue figure indicates the number of failed transmissions.
2. Data that was not transmitted can be resent from the memory. See [section 6.1.2 Transmitting and deleting stored recordings, page 64](#).

If no data can be transmitted, check the following:

- Network settings (see [page 78](#))
- Network connection WLAN or LAN
- Encryption settings on the server
- Settings in the Schiller Link.

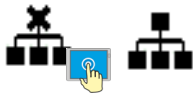

Connectivity ⌵

Ethernet	N/A	[MAC -----]
Wi-Fi	N/A	[MAC -----]
EPA-Integration	Schiller-Server	

URL not valid

Close

10 Maintenance



The regular system maintenance must include a software check according to the manufacturer's instructions. The test results must be recorded and compared to the values in the accompanying documents.

Maintenance work not described in this section may only be accomplished by a qualified technician authorized by Spacelabs Healthcare.

The following table indicates the intervals and responsibilities of the maintenance work required. Local regulations in your country may stipulate additional or different inspection intervals and tests.

Interval	Maintenance step	Responsible
Before each use	<ul style="list-style-type: none"> • Visual inspection of the device and ECG electrodes 	→ User
Every 6 months	<ul style="list-style-type: none"> • Visual inspection of the device (see page 93, 10.6 Inspection report) <ul style="list-style-type: none"> – Touch screen test – Cables and accessories – Power supply unit and AC power cable • Functional tests according to the instructions (see page 93, 10.6 Inspection report) 	→ User
Every 12 months	<ul style="list-style-type: none"> • Safety test according to IEC/EN 62353 	→ Qualified service personnel

10.1 Visual Inspection

Visual inspection of the unit and the cable assembly for the following:

- Device casing and power supply unit not broken or cracked
- LCD screen not broken or cracked
- Electrode cable sheathing and connectors not damaged
- No kinks, abrasion or wear in any cable assembly.
- Input/output connectors not damaged.

In addition to the visual inspection, switch on the CardioPulse Go, toggle through the menu and check some functions randomly. In this way, you can check that:

- the device performs faultlessly
- the display works
- the touch screen works



- ▲ Defective units or damaged cables must be replaced immediately.

10.2 Cleaning the casing and cables

WARNING

- ▲ Switch the device off before cleaning and disconnect it from the AC power by removing the plug. Do not, under any circumstances, immerse the device in cleaning liquid and do not sterilize it with hot water, steam or air.

CAUTION

- ▲ Do not autoclave the device and accessories.
- ▲ Do not immerse the device in liquid.
- ▲ Do not spray liquid onto the device/cable.
- ▲ The use of detergents with a high acid content or detergents that are otherwise unsuitable can damage the device (i.e. cracks and wear of the plastic casing).
- ▲ Always follow the usage instructions provided by the manufacturer of the cleaning solution.
- ▲ With time, the casing may become less resistant:
 - if an alkaline cleaner or a cleaner with a high alcohol concentration is left for a long time on the surface, or
 - if a warm disinfectant or detergent is used. Spacelabs Healthcare therefore recommends using only cleaning agents that are adequate for sensitive materials such as plastics, and using them at room temperature (approx. 68 °F).
- ▲ Never use any of the following solutions or similar products to clean the equipment: ethyl alcohol, acetone, hexane, abrasive or scouring powder or material, any cleaning material that damages plastic.
- ▲ The patient cable and other cable assemblies must not be exposed to excessive mechanical stress. Whenever disconnecting the leads, hold the plugs and not the cables. Store the leads in such a way as to prevent anyone stumbling over them or any damage being caused by the wheels of instrument trolleys.
- ▲ When cleaning, ensure that all labels and safety statements, whether etched, printed or stuck to the device, remain in place and remain readable.


Thoroughly inspect the device and the accessories before cleaning.

- Look for any signs of damage and make sure that the keys and connectors work correctly.
- Gently bend and flex cables, inspecting them for damage or extreme wear, exposed wires, and bent connectors.
- Confirm that all connectors engage securely.


The casing of the CardioPulse Go and the cable assemblies can be cleaned with a cloth slightly moistened (not wet) on the surface only. If necessary, a domestic non-caustic cleaner or a 50 % alcohol solution can be used to remove grease stains and finger prints. Wipe the equipment with a cloth slightly moistened (not wet) with one of the approved cleaning solutions (see section 10.2.2). Thoroughly wipe off any excess cleaning solution. Do not let the cleaning solution run into or accumulate in connector openings, switches, or gaps. If liquid gets into connectors, dry the area with warm air and check that the device operates properly.

10.2.1 Cleaning the cable assembly

1. Before cleaning, inspect the cable for damage. Gently bend and flex all parts of the cable. Inspect for splits in the sheathing, damage or extreme wear, exposed wires or bent connectors.
2. Wipe the cable with a cloth slightly moistened (not wet) with one of the approved cleaning solutions listed below.
3. Gently grip the cable with the damp cloth in the center of the cable and slide the cable through the cloth 7.9 inches at a time until clean. Do not clean the whole length in one single action as this may cause 'bunching' of the insulation sheathing.

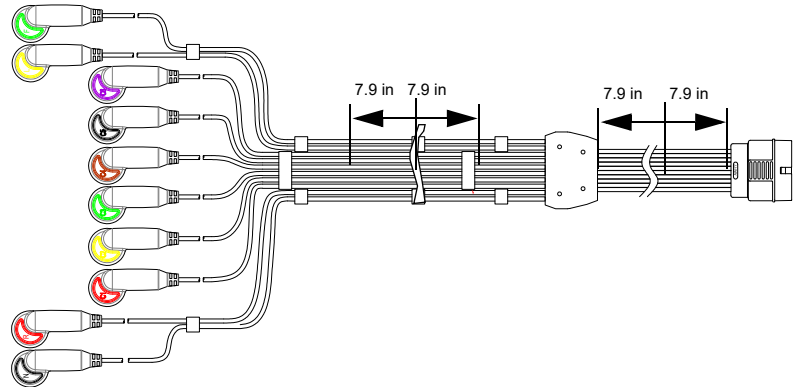

4. Thoroughly wipe off any excess cleaning solution. Do not let the cleaning solution run into or accumulate in connector openings, switches, or gaps. If liquid gets into connectors, dry the area with warm air.

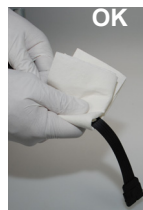

10.2.2 Approved cleaning solutions

- 50 % isopropyl alcohol
- neutral, mild detergent
- all products designed for cleaning plastic.

10.2.3 Non-admissible detergents

Never use products containing the following:

- Ethyl alcohol
- Acetone
- Hexane
- Abrasive cleaning powder
- Plastic-dissolving products

10.3 Disinfection

Disinfection removes certain bacteria and viruses.

Use commercially available disinfectants intended for clinics, hospitals and medical practices. Please refer to the information of the manufacturer's disinfectants for the application.

Disinfect the units in the same way as described for cleaning the units ([previous page](#)).

10.3.1 Approved disinfectants

- Isopropyl alcohol 50%
- Propanol (35 %)
- Aldehyde (2-4 %)
- Ethanol (50 %)
- all products that are suitable for sensitive surfaces, such as:
 - Bacillol® 30 foam/ Bacillol® 30 Tissues (10% Propanol-1, 15 % Propanol-2, 20 % Ethanol)
 - Mikrozyd® AF (25 % Ethanol, 35 % 1Propanol-1)

10.3.2 Non-admissible disinfectants

Never use products containing the following:

- Organic solvents
- Ammonia-based detergent
- Abrasive cleaning agents
- 100 % alcohol
- Conductive solution
- Solutions or products with the following ingredients:
 - Acetone
 - Quaternary ammonium compound
 - Betadine
 - Chlorine, wax or wax compound
 - Ketone
 - Sodium salt

10.4 Cleaning the thermal print head



Over a period of time, the printing ink from the grid on the paper can form a film on the thermal print head. This can cause the print quality to deteriorate. We recommend therefore that the print head is cleaned with alcohol every month. This is done as follows:

1. Open the paper tray and remove the paper. The thermal print head is located directly above the pressure roller (when the paper tray is closed).
2. With a tissue dampened in alcohol, gently rub the printhead to remove the ink residue. If the print head is badly soiled, the color of the paper grid ink will show on the tissue.


10.5 Battery

- No maintenance is required for the lithium-ion polymer batteries.
- Replace the battery approximately every 4 years (depending on the application) when the battery running time falls substantially under one hour.
- Storage and operation conditions outside the temperature range of 59 - 77 °F will reduce the service life of the battery!
- Make sure that the batteries remain charged during storage. If the device is not used for more than 3 to 4 months, the battery needs to be protected from deep discharge by recharging it; the ideal capacity is 50-80%. If a fully charged battery is stored for a long period of time, this may reduce its service life.

10.5.1 Charging the battery

A totally discharged battery requires approximately 3 hours to be 100% charged (when the unit is switched off). It is possible to use the unit when the battery is being charged; however, the charging time may be longer.

No harm will be done to the battery by leaving the unit connected to the AC power supply.

1. Connect the device to the AC power supply.
2. AC power via external power supply unit .
3. The blinking battery LED indicates that the battery is being charged.
4. Charge the battery for at least 3 hours.

10.5.2 Battery disposal



The battery must be disposed of in municipally approved areas or sent back to Spacelabs Healthcare.

WARNING

- ▲ Explosion hazard! The battery must not be burned or disposed of in domestic waste.
- ▲ Acid burn hazard! Do not open the battery.

10.6 Inspection report



- ▲ The user guide, especially chapter 10, must be read before the inspection.
- ▲ **Recommended inspection interval:** Every 6 months

Serial no.: _____

Test	Results	Date				
Visual inspection 10.1	• Casing not damaged	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
→ External condition						
	• Electrode connector port not damaged	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
→ Availability and condition of accessories	• ECG Electrodes (expiration date and compatibility)	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	• User Guide	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	• AC power and patient cable	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Functional test 2.3	• No error message shown in the standard display	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
→ ECG test						
→ Multi-touch screen test	• Touch screen works	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
→ Check the battery	• Battery OK	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
→ Printer	• Contrast and line strength	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	• Cleaning the thermal print head	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Remarks						
→ Recurrent test conducted (every 12 months)			<input type="checkbox"/>		<input type="checkbox"/>	
Inspection carried out by:						

In case of a defect, please contact the service department of your hospital , your Spacelabs Healthcare representative or the local after-sales service .

Name:


Phone:


10.6.1 Lifer-item replacement every 3 - 5 years

Inspection	Results	Replacement				
Internal battery						
→ Replace internal battery if operation falls substantially under one hour.	• Unit sent to Spacelabs Healthcare service center for accumulator replacement.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	Date of replacement:					
	Inspector:					

11 Trouble shooting

11.1 Possible problems

Error	Possible causes and indicators	Error localization and troubleshooting
Unit does not switch on; blank screen	<ul style="list-style-type: none"> No AC power supply; green AC power indicator on the power supply unit is off. 	<ul style="list-style-type: none"> → Check the power supply unit and fuses. → If the AC power indicator is lit, it indicates that power is reaching the unit and the internal power supply should be OK. Press and hold the On/Off key for 10 seconds. Wait a few seconds and switch the device on again.
	<ul style="list-style-type: none"> AC power connection OK, but indicator  and LED are not lit. 	<ul style="list-style-type: none"> → Check / change the battery. If the battery is faulty, it is possible that the unit cannot be switched on even if the AC power supply is connected. → If the screen is still not lit, it indicates a software fault, monitor problem or internal power supply. Call your local Spacelabs Healthcare representative.
QRS traces overlap	<ul style="list-style-type: none"> Incorrect settings for patient. 	<ul style="list-style-type: none"> → Change sensitivity setting. → Check electrode contact and reposition the electrodes.
	<ul style="list-style-type: none"> Bad electrode contact. 	<ul style="list-style-type: none"> → If the problem persists, call your local Spacelabs Healthcare representative. → Note: Some patients have very high amplitudes and even on the lowest sensitivity settings, the QRS traces can overlap.
"Noisy" traces	<ul style="list-style-type: none"> High resistance between skin and electrodes 	<ul style="list-style-type: none"> → Check the electrode resistance (all leads need to be shown in green) → Re-apply electrodes.
	<ul style="list-style-type: none"> Patient not relaxed 	<ul style="list-style-type: none"> → Ensure that the patient is warm and relaxed.
	<ul style="list-style-type: none"> Incorrect settings 	<ul style="list-style-type: none"> → Check all filter settings (Menu > Settings > ECG > Filters & formulas). → Activate the Myogram (Low pass) filter and change the cut-off frequency. → Ensure mains filter is correct for AC power supply.
		<ul style="list-style-type: none"> → If the problem persists, call your local Spacelabs Healthcare representative.
No printout obtained after an auto mode recording.	<ul style="list-style-type: none"> No paper 	<ul style="list-style-type: none"> → Ensure that paper is loaded.
	<ul style="list-style-type: none"> Paper incorrectly loaded. 	<ul style="list-style-type: none"> → Reload paper. → Ensure that the paper has been inserted correctly with the black mark at the top.
	<ul style="list-style-type: none"> Incorrect settings 	<ul style="list-style-type: none"> → Check that the printout is activated for at least one setting, and that Print after acquisition is activated (see page 58 and 80)
		<ul style="list-style-type: none"> → If the problem persists, call your local Spacelabs Healthcare representative.

Error	Possible causes and indicators	Error localization and troubleshooting
	<ul style="list-style-type: none"> • Old paper inserted. 	<ul style="list-style-type: none"> → Ensure that new Spacelabs Healthcare paper is inserted. → Note that the CardioPulse Go thermal paper is heat- and light-sensitive. If it is not stored in its original seal, stored in high temperatures or is simply old, print quality can deteriorate.
Printout fades, is not clear or the printout is 'patchy'.	<ul style="list-style-type: none"> • Dirty print head • Line width/contrast setting incorrect • Print-head out of adjustment. 	<ul style="list-style-type: none"> → Over a period of time, the printing ink from the grid on the paper can form a film on the thermal print head. Clean the thermal print head. → Set the line width, contrast, printer, see page 82. → If the problem persists, call your local Spacelabs Healthcare representative.
No printout of interpretation statement, average cycles or measurements	<ul style="list-style-type: none"> • Incorrect settings • Software hangs up 	<ul style="list-style-type: none"> → Check that the interpretation and measurement options are enabled for the printout and that the lead sequence is set to Normal (see page 58 section 5.6.7 and page 41 section 4.11.2, → Switch off and on again after a few seconds. → Press and hold the On/Off button for 10 seconds to force the device to switch off. Reconnect AC power and switch on. → If the problem persists, call your local Spacelabs Healthcare representative.
Multi-touch screen is blocked		
Multi-touch screen responds in an irregular way	<ul style="list-style-type: none"> • Excessive EMC interferences 	<ul style="list-style-type: none"> → Check for sources of excessive EMC interferences.
Not able to start ECG recording	<p>Memory full</p> 	<ul style="list-style-type: none"> → Delete old ECG recordings, see page 62.

11.2 Preventing electromagnetic interferences



"Non ionising electromagnetic radiation"

The user can help avoid electromagnetic disturbances by keeping the minimum distance between **portable** and **mobile** HF telecommunication devices (transmitters) and the CardioPulse Go. The distance depends on the output performance of the communication device as indicated below.

HF source Wireless communications devices	Transmitter frequency [MHz]	Testing frequency [MHz]	Max. power P [W]	Distance d [m]
Various radio services (TETRA 400)	380-390	385	1.8	0.3
- Walkie-talkies (FRS) - Rescue service, police, fire brigade, servicing (GMRS)	430-470	450	2	0.3
LTE band 13/17	704-787	710/745/780	0.2	0.3
- GSM800/900 - LTE band 5 - Radio telephone (microcellular) CT1+, CT2, CT3	800-960	810/870/930	2	0.3
- GSM1800/1900 - DECT (radio telephone) - LTE Band 1/3/4/25 - UMTS	1700-1990	1720/1845/ 1970	2	0.3
- Bluetooth, WLAN 802.11b/g/n - LTE Band 7 - RFID 2450 (active and passive transponders and reading devices)	2400-2570	2450	2	0.3
WLAN 802.11a/n	5100-5800	5240/5500/ 5785	0.2	0.3



- ▲ **Portable** HF telecommunication devices must not be used within a radius of 0.3 m from the CardioPulse Go and its cables.
- ▲ Do not place the CardioPulse Go on top of other electric/electronic devices - i.e. maintain a sufficient distance to other devices (this includes the patient cables).

For permanent HF telecommunication devices (e.g. radio and TV), the recommended distance can be calculated using the following formula: $d = 0.6 \times \sqrt{P}$. (The formula is based on the max. immunity level of 10 V/m in the frequency domain of 80 MHz to 3000 MHz).

d = recommended minimum distance in meters
P = transmitting power in Watts



For more information on operation in an electromagnetic environment according to IEC/EN 60601-1-2 see [section For fixed installed HF-Transmitters \(for example, Radio and TV transmitters\)](#) the following minimum distance to the transmitter can be calculated as follows:., page 108.

11.2.1 Measures to prevent electromagnetic interferences

The user can take the following measures to prevent electromagnetic interferences:

- Increase distance to the source of interference.
- Turn the device to change the angle of radiation.
- Connect the potential equalisation cable.
- Connect the device to a different AC power connector.
- Only use original accessories (especially patient cables).
- Immediately replace defective cables, especially patient cables with defective sheathing.
- Make sure the patient cable is securely screwed on.
- Observe the maintenance intervals as stated in [section 10 Maintenance, page 88](#).

11.3 Accessories and disposables



▲ Always use Spacelabs Healthcare replacement parts and disposables, or products approved by Spacelabs Healthcare. Failure to do so may endanger life and/or invalidate the warranty.

Your local representative Spacelabs Healthcare stocks all the disposables and accessories available for the CardioPulse Go. A comprehensive list of all Spacelabs Healthcare representatives can be found on the Spacelabs Healthcare website (www.spacelabshealthcare.com). In case of difficulty, contact our head office. Our staff will be pleased to help process your order or to provide information on all Spacelabs Healthcare products.

11.3.1 ECG Cable Patient cables

REF no.:	CAT no.:	Label	Description
2.400175	-	ECG D-SUB C 4.6/4.25 x10 IEC	10-wire patient cable IEC, clip type, 4.6/4.25 m
2.400178	-	ECG D-SUB C 4.6/4.25 x10 AHA	10-wire patient cable AHA, clip type, 4.6/4.25 m
2.400179	-	ECG D-SUB B 3.1/2.55 x10 AHA	10-wire patient cable AHA, banana plug, 3.1/2.55 m
2.400180	-	ECG D-SUB B 3.1/2.55 x10 IEC	10-wire patient cable IEC, banana plug, 3.1/2.55 m
2.400226	-	ECG FS P 2.1/1.6 x10 IEC	10-wire patient cable, IEC, push-button, 2.1/1.6 m
2.400227	700-0386-00	ECG FS P 2.1/1.6 x10 AHA	10-wire patient cable, AHA, push-button, 2.1/1.6 m
2.400330	-	ECG FS B 2.1/1.6 x10 IEC	10-wire patient cable, IEC, banana plug, 2.1/1.6 m
2.400331	00-0387-00	ECG FS B 2.1/1.6 x10 AHA	10-wire patient cable, AHA, banana plug, 2.1/1.6 m

11.3.2 Electrodes and ECG consumables

REF no.:	CAT no.:	Label	Description
2.000041	-	ECG KIT	Electrode kit for adults (metal) including electrode gel
2.000052	-	ECG KIT	Electrode kit for children (metal) including electrode gel
2.155020	-	Set of 4 clamp electrodes	Limb electrodes, adults (metal)
2.155000	-	Suction chest electrode	Suction electrodes chest, 24 mm, set of 6 pieces
2.310317	-	ECG FS D-SUB 0.25 x10 AC	Suction pump adaptor
2.155025	-	Ambu® BlueSensor R	Blue Sensor electrodes for exercise ECG (500 pieces)
2.155032	-	Ambu® Snap Clip	Adapter for banana plugs (10 pieces)
2.155034	-	Ambu® WhiteSensor 0415M	White Sensor electrodes for resting ECG (500 pieces)
2.157055	010-2281-00	ECG Recording Paper Fanfold	Thermal chart paper, box of 10 (60 sheets each)

11.3.3 AC power cable, Power supply and potential equalisation

REF no.:	CAT no.:	Article
2.310320	161-0280-00	Earth cable for the potential equalisation stud
2.300000	-	AC power cable Switzerland, straight, 2.5m
2.300002	-	AC power cable Schuko Europe, 90° angled, 2.5m
2.300003	-	AC power cable Switzerland, 90° angled, 2.5m
2.300004	-	AC power cable UK, 90° angled, 2.5m
2.300005	-	AC power cable Schuko Europe, straight, 2.5m
2.300011	-	AC power cable UK, straight, 2.5m

REF no.:	CAT no.:	Article
2.300012	-	AC power cable USA (hospital grade), straight, 2.5m
2.300014	-	AC power cable China, 90 ° angled, 2.5m
2.300016	-	AC power cable Japan, 90 ° angled, 2.5m
2.300024	161-0278-00	AC power cable USA (hospital grade), 90 ° angled, 2.5m
2.300025	-	AC power cable Brazil, 90 ° angled, 2.5m
2.200136	161-0281-00	Medical-grade power supply unit, 100-240 VAC, 1.0 -0.6 A, 50/60Hz, 15 VDC 30 Watts (replacement of the 2.200126)

11.3.4 Other accessories

REF no.:	CAT no.:	Article
2.310317	700-0399-00	D-SUB to FT-1 ECG cable connector adapter
2.000147	-	Barcode scanner set including barcode scanner (2.200208 // LS2208-SR20001R-UR) and SCHILLER User guide "Barcode scanner for use with SCHILLER units" (2.510721)

11.3.5 Trolley

Art. no.:	Label	Article
2.101126	Trolley X1 with console for Cardio Pulse	Trolley X1 incl. basket and mounting bracket for CardioPulse Go
2.101118	Drawer for trolleys X1, X2, X3, X4	Drawer
2.101119	Basket translucent for trolleys X1, X2, X3, X4, X5	Transparent basket
2.101121	Holder barcode scanner for trolleys X1, X2, X3, X4, X5	Barcode scanner holder
2.101130	ECG patient cable support for trolleys X1, X3	ECG patient cable support
2.101181	Holder Strässle for DT-80 for X1	Holder for Strässle suction pump DT-80
2.101184	Holder Strässle for DT-100 for X1	Holder for Strässle suction pump DT-100
2.101143	Kit screws for trolley X1	Replacement screw kit for trolley
2.101163	HOLDER FOR X1 TROLLEY	Holder for accessories
2.101174	Wheel with braking function for X1	Wheel with brake

12 Technical data

12.1 Device

Dimensions	230 x 160 x 30 mm, approx. 1.1 kg incl. thermal paper
Display	<ul style="list-style-type: none"> Multi-touch Backlit LCD screen for graphic and alphanumeric representation Resolution: 1200 x 768 dots, 8 "
Power supply with:	Medical grade switching power supply, protection class I
<ul style="list-style-type: none"> external power supply unit 	(Note: because the protective earth (PE) ends in the power supply, the protective earth test does not need to be performed.)
Input	100 - 240 VAC, max. 1.0 A (100 V) - 0.6 A (240V), 50-60 Hz
Output to CardioPulse Go	15 VDC, max. 2 A
<ul style="list-style-type: none"> Battery 	Operation with built-in rechargeable battery
Power supply device	15 VDC 30 VA
Battery	
Capacity	<ul style="list-style-type: none"> Lithium-ion polymer 11.1 V, 2.4 Ah 4 hours normal use without printing and WLAN
Battery life	Under normal operating conditions, 4 years
Charging time	100 %: approx. 3 hours when the device is switched off
Printer	High-resolution thermal head printer; 8 dots/mm (amplitude axis); 20 dots/mm (time axis) @ 25 mm/s
Frequency range	0 to 250 Hz (IEC/AHA)
Chart paper	Thermo-reactive, Z-fold, 114 mm wide, optimal positioning on 150 mm
Speed	<ul style="list-style-type: none"> 5/12.5/ 25/ 50 mm/s (5 /12.5 mm/s only available for manual printouts), for resting rhythm: 6.25 mm/s
Sensitivity	<ul style="list-style-type: none"> 5 /10 / 20 mm/mV, for resting rhythm: 2.5/5 mm/mV
ECG review	Display of the ECG on an area of 118 x 192 mm with different layouts.
Speed	<ul style="list-style-type: none"> 12.5 / 25 / 50 mm/s, for resting rhythm: 6.25 mm/s
Sensitivity	<ul style="list-style-type: none"> 5 /10 / 20 mm/mV, for resting rhythm: 2.5/5 mm/mV
Interfaces	<ul style="list-style-type: none"> ECG cable connection Potential equalisation Network connection (1Gbit) 2x USB type A 1x USB type B is not intended for the US market
Memory	Memory for at least 350 ECG recordings and 100 resting rhythm recordings

Ambient conditions

Operating temperature	• 50 to 104 °F
Relative humidity	• 15 to 95% (non-condensing)
Pressure during operation	• 700 to 1060 hPa
Storage temperature	• 41 to 122 °F
Transport temperature	• 14 to 122 F
Humidity during storage/transport	• 10 to 95% (non-condensing)
Pressure during storage/transport	• 500 to 1060 hPa

12.2 ECG

Patient input	Fully floating and isolated, defibrillation protected (only with original Spacelabs Healthcare patient cable)
Lead configurations	<ul style="list-style-type: none"> • Standard 12-lead • Paediatric • Right precordials • Standard C4r • Left posterior • Nehb • Balanced
Display	
Leads	<ul style="list-style-type: none"> • Display of the selected leads, 2x6 or 4x3 leads (configurable) <ul style="list-style-type: none"> – Chart speed 12.5, 25 or 55 mm/s (configurable) – Amplitude 5, 10 or 20 mm/V (configurable)
Status	<ul style="list-style-type: none"> • Filter status • Power source • Leads • Electrode contact status • Heart rate (HR) • Date and time • Patient name and number • WLAN transmission
Filter	
Myogram filter (muscle tremor)	Set to 25, 40, 150, 250 Hz (250 Hz = Filter Off)
Notch filter	Distortion-free suppression of superimposed AC 50 or AC 60 Hz sinusoidal interferences by means of adaptive digital filtering
Data record	<ul style="list-style-type: none"> • Patient data (name, age, height, weight, BP, device ID, MTA ID) • Listing of all ECG recording conditions (date, time, filter) • ECG measurement results (intervals, amplitudes, electrical axes) • Average complexes • Guidance on interpreting adult and pediatric ECGs
With optional interpretation (C) program	
ECG amplifier	Complies with IEC 60601-2-25 and ANSI/AAMI EC11

12.3 Safety standards

Safety standard

IEC/EN 60601-1
IEC/EN 60601-2-25

EMC

IEC/EN 60601-1-2

Protection class

Device as a system: Class I in accordance with IEC/EN 60601-1

Conformity/classification

CE/IIa in accordance with directive 93/42/EEC

Protection

This device is not designed for outdoor use (IP 20)

12.4 WLAN standards

Modules

WL1837MOD

FCC ID
IC ID

Z64-WL180DBMOD
4511-WL18DBMOD

Transmission standards

IEEE 802.11 a, b, g, n

Safety/encryption

WPA2 enterprise /ieee802.1, WPA2-PSK, WPA-PSK, WEP64/128/256, TKIP, AES

Frequency range

Dual-band 2.4 GHz and 5 GHz

Max. power output 2.4 GHz (1DSSS)

+16.5 dBm

Max. power output 5 GHz (OFDM6)

+18 dBm

12.5 EMC information

The unit meets the Collateral Standards of Electromagnetic compatibility – Requirements and tests IEC/EN 60601-1-2 the limits and methods of measurement of electromagnetic disturbance characteristics of industrial, scientific and medical radio frequency equipment.

Medical electrical equipment is subject in regard to the electromagnetic compatibility (EMC) and its special precautionary measure.

The unit must in reference to the mentioned EMC-hints in the accompanying documents be installed and operated.

This medical device is intended for use in the electromagnetic environment specified in the following tables. The user of this device should ensure that it is used in such an environment.

12.5.1 Electromagnetic emissions


Emission	Test Compliance	Electromagnetic environment guidance
RF emissions CISPR 11/32	Group 1	This device uses RF energy only for its internal function. Therefore its RF emission are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/Flicker emissions IEC 6100-3-3	Complies	

12.5.2 Immunity

Immunity Test	IEC 606101 Test level	Compliance Level	Electromagnetic environment guidance
ESD IEC 61000-4-2	±8kV Contact ±15kV Air	±8kV Contact ±15kV Air	Floors should be wood, concrete or ceramic tile. If floors are synthetic, the relative humidity (RH) should be at least 30%.
EFT IEC 61000-4-4	±2kV Power supply lines ±1kV I/O lines	±2kV Power supply lines ±1kV I/O lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1kV differential mode ±2kV common mode	±1kV differential mode ±2kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage Dips/Dropout IEC 61000-4-11	< 5% U_T (> 95% dip in U_T) for 0,5 cycle 40% U_T (60% dip in U_T) for 5 cycles 70% U_T (30% dip in U_T) for 25 cycles < 5% U_T (> 95% dip in U_T) for 5 s	< 5% U_T (> 95% dip in U_T) for 0,5 cycle 40% U_T (60% dip in U_T) for 5 cycles 70% U_T (30% dip in U_T) for 25 cycles < 5% U_T (> 95% dip in U_T) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. The unit shutoff during the >95% for 5 second disturbance. If the user of the HT300-series meter requires continued operation during power mains interruptions, it is recommended that the HT300-Series meter be powered from an uninterrupted power supply or battery.
Power Frequency 50/60Hz Magnetic Field IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be that of a typical commercial or hospital environment.

NOTE U_T is the AC mains voltage prior to application of the test level.

12.5.3 Emissions equipment and systems

Immunity Test	IEC 606101 Test Level	Compliance Level	Electromagnetic environment guidance
			Portable and mobile communications equipment should be used no closer to any part of this device, including cable, than the recommended separation distance (d) calculated from the equation applicable to the frequency of the transmitter Recommended separation distance:
Conducted RF IEC 61000-4-6	3 Vrms outside ISM band 6 Vrms in the ISM & amateur radio band 150 kHz to 80 MHz	$[V_1] = 10 \text{ Vrms}$	$d = \frac{3.5}{V_1} \times \sqrt{P}$
Radiated RF IEC 61000-4-3	10 V/m 80 MHz to 2700 MHz	$[E_1] = 10 \text{ V/m}$ 80 MHz to 3000 MHz	$d = \frac{6}{E_1} \times \sqrt{P}$ for 80 MHz to 3000 MHz
Proximity fields from RF wireless communications equipment IEC 61000-4-3	see section 12.5.4 Immunity to proximity fields from RF wireless communications equipment, page 107	section 12.5.4 Immunity to proximity fields from RF wireless communications equipment, page 107	$d = \frac{6}{E_1} \times \sqrt{P}$ for tested frequencies Tested Frequency see section 12.5.4 Immunity to proximity fields from RF wireless communications equipment, page 107
			where P is the maximum power in watts and d is the recommended separation distance in meters. Field strengths from fixed transmitters, as determined by an electromagnetic site ^a survey, should be less than the compliance ^b levels (V_1 and E_1). Interference may occur in the vicinity of equipment marked with following Symbol  "non ionizing radiation"

Note 1 At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To access the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocation the device.

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than $[V_1]$ V/m.

12.5.4 Immunity to proximity fields from RF wireless communications equipment

Test frequency [MHz]	Band ^a [MHz]	Service	Modulation	max. power P [W]	Distance d [m]	Immunity level [V/m]
385	380-390	Various Transmitter devices (TETRA 400)	Pulse modulation ^b 18 Hz	1.8	0.3	27
450	430-470	- Walkie-talkie (FRS) - Emergency services, Police, Fire, Maintenance (GMRS)	FM ^c ±5 KHz ±1 KHz sine	2	0.3	28
710 745 780	704-787	L TE Band 13/17	Pulse modulation 217 Hz	0.2	0.3	9
810 870 930	800-960	- GSM800/900 - LTE band 5 - Funktelefon CT1+, CT2,CT3	Pulse modulation 18 Hz	2	0.3	28
1720 1845 1970	1700-1990	- GSM1800/1900 - DECT (mobile telephone) - LTE Band 1/3/4/25 - UMTS (Universal Mobile Telecommunications System)	Pulse modulation 217 Hz	2	0.3	28
2450	2400-2570	- Bluetooth, WLAN 802.11b/g/n - LTE Band 7 - RFID 2450 (active & passive transponder and readers)	Pulse modulation 217 Hz	2	0.3	28
5240 5500 5785	5100-5800	WLAN 802.11a/n	Pulse modulation 217 Hz	0.2	0.3	9

a. For some services, only the uplink frequencies are included.

b. The carrier shall be modulated using a 50 % duty cycle square wave signal.

c. As an alternative to FM modulation, 50 % pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case

12.5.5 Recommended separation distances

For fixed installed HF-Transmitters (for example, Radio and TV transmitters) the following minimum distance to the transmitter can be calculated as follows:

Maximum Power Output [Watts]	Separation distance according frequency of the transmitter [m]	
	150 kHz to 80 MHz	80 MHz to 3000 GHz
	$d = \frac{3.5}{V_1} \times \sqrt{P}$	$d = \frac{6}{E_1} \times \sqrt{P}$
0.01	0.04	0.06
0.1	0.11	0.19
1	0.35	0.6
10	1.1	1.9
100	3.5	6

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts [W] according to the transmitter manufacturer.

Note 1 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.










Blank page


















13 Index




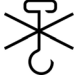

A		N	
Accessories and disposables	99, 108	Nehb leads	39
B		Network connection	16
Baseline filter	51	Notch filter	51
Battery		O	
Battery life	101	Operation – Overview	18
Capacity	101	Options	16
Charging time	101	P	
Battery operation	23	Patient data - entering	26
C		Potential equalisation	21
Cabrera lead group	47, 50	Power supply	23
Cabrera lead sequence - settings	41	R	
Celsius	79	Receiving a Joblist	65
Cleaning	89	Resting ECG	42
Communication		Automatic Mode Recording	45
Defining WLAN	83	Automatic printout	46
Connections	20	Lead group	50
E		Manual printout	47, 50
Electrodes		Resting ECG - Procedural Flow	
Colour code	32	Diagram	43
Electrode and patient cable check		S	
(lead test)	40	Safety Notes	7
Placement	31	Sequential	58, 75
Placement for standard 10-lead resting		Signal averaged ECG	62
ECG	33	Simultaneous	58, 75
Skin/Electrode Resistance	40	Standard Lead Group	47, 50
Emergency ECG at startup	22	Standard lead sequence - settings	41
F		Switching on / off	22
Fahrenheit	79	T	
I		Transmission with pacemaker patient	83
Isolating the device from the mains		W	
supply	23	Worklist	
L		Performing a worklist recording	67
Lead sequence	41	Receiving a Joblist	65
M		Sending worklist recordings to the HIS	69
Main	15		
Maintenance	88		
Myogram filter (Low Pass)	51		

14 Appendix – Symbols

This appendix lists all general symbols that may be present on the device, label and accessories. Not all of those symbols are necessarily present on your device.

	Identification of the manufacturer
	Identification of the manufacturing date
	Identification of the distributor
	Identification of the importer
MD	Medical device
SN	Serial number
REF	Reference number
LOT	Batch code
GTIN	Global Trade Item Number
CAT	Catalogue number
QTY	Quantity
UDI	UDI: unique device identification as QR code machine readable and human readable as number (e.g.  (01) 0 7613365 00210 2 (21)xxxx.xxxxxx)
	Number of pieces in the packaging
EC REP	Authorised European representative
	Notified body (e.g.  0123 marking notified body TÜV SÜD)
	CE marking, affirms its conformity with European standards

	Regulatory Compliance Mark for the Australian standards
	The device is recyclable
	Symbol for the recognition of electrical and electronic equipment. Device must not be disposed of in the household waste.
	Symbol for the recognition of a battery. Battery must not be disposed of in the household waste.
	The packaging is made in low density polyethylene and can be recycled.
	Federal law (USA) restricts this device to sale by or on the order of a physician
	Non ionising electromagnetic radiation. To indicate that the device contains a Radio Frequency (RF) transmitter to transmit data (e.g Bluetooth or WiFi)
	Contains a Bluetooth module
	Do not reuse
	Latex-free
	Use-by date (expiry date of battery, electrodes or other consumables)
	Temperature range for storage or transport, respectively
	Pressure range for storage or transport, respectively
	Humidity range for storage or transport, respectively
	Consult instruction for use (indicates the need for the user to consult the instructions for use)
	Use within X days after opening (electrodes or other consumables)
	Keep dry (store in a dry location)

	Keep away from sunlight (protect from direct sunlight)
	Fragile, handle with care
	Transport upwards (this way up)
	Do not use hooks
	EIP = electronic information product (does not contain any toxic and hazardous substances or elements above the maximum concentration values (product can be recycled and re-used).



Blank page