

Spacelabs Healthcare produces clinically beneficial, cost-effective monitoring and clinical information systems. As part of our commitment to the continual development of our product line, we have released the latest version of the product listed below. This bulletin is designed to inform you of the new features or enhancements. Thank you for your continued support.

Update specifications

Updated System Components	SafeNSound Mobile 96180
Software Version	5.0

New features or enhancements

SafeNSound™ Mobile 5.0 contains a general rebuild of the mobile application with limited functionality in this initial release. This version offers support for XprezzNet 2.0.1+ (96190) and SAI 1.1, as well as minor improvements mentioned below.

Mobile updates will be distributed by the hospital organization's preferred choice of MDM Solution. This should be managed by the organization.

Bugs

- XprezzNet 2.0.1+ (96190) - Supports XprezzNet 2.0.1+ and is compatible with SafeNSound's subsequent Web App version of 5.9.0+.
- Smart Alarm Interface 1.1 - Modified to utilize and support SAI 1.1 functionality.
- Communication - Improved overall functionality of Voice Calls and Quick Messages.
- Bedside - Refined general performance of Bedside for the purpose of admitting, transferring, and discharging patients.
- Patient Data - Removed the ability to select a patient and view corresponding patient data in this initial release. Remaining functionality will be restored in the next release.
- Patient Dashboard - Live Waveforms on the Dashboard are disabled. However, Live Vitals remain intact and update in near-real time.
- Also Disabled the following tools: Messaging, Profile, Support.

Supported versions

- Web client version 8.0+