

Spacelabs Healthcare produces clinically beneficial, cost-effective monitoring and clinical information systems. As part of our commitment to the continual development of our product line, we have released the latest version of the product listed below. This bulletin is designed to inform you of the new features or enhancements. Thank you for your continued support.

Update specifications

Updated System Components	SafeNSound 96180
Software Version	5.9.4

New features or enhancements

SafeNSound™ software version 5.9.4 is a release which includes only minor bug fixes.

Expected downtime for this update will vary depending on what version customers are currently on. Users will be logged off and required to log in once the update has been completed. For enterprise customers, please email devops@spacelabs.com to request to be updated to this latest version.

Other changes and updates

- ADT Message Search - Improved speed of patient ADT search when scanning in Bedside.
- End Call Delay - Resolved delayed response when attempting to end a call.
- Call Connection Delay - Resolved second delay before call is connected.
- Incoming Call Audio - Resolved stacking of call audio files in the Web App when receiving multiple incoming calls at once.
- Username on Dashboard - Resolved incorrect username displaying on Dashboard.
- Server Communication Performance - Internal communication improvements between services, resulting in higher performance
- Default Parameter Selection - Default waveform parameters within the Details tab are automatically checked if the event leads are disconnected and reconnected to a patient.
- Patient Timeline Entries - Erroneous entries related to clearing order message notes no longer appear in the patient Timeline.
- Saved Event Selection - Resolved inability to close Saved Event with X button if the user double-clicked to open.
- Twilio Application Errors - Resolved 401 application error occurring on incoming and outbound calls.

- Waveforms - Overall improvements and stability around waveform processing

Supported versions

- Web client version 8.0+
- Mobile version 5.0+

Rollback plan

The rollback plan is dependent on your current version of SafeNSound (SNS) and XprezzNet (XN):

- Customers on XN 1.3.6 and associated versions of SNS would be rolled back to XN 1.3.6 and SNS 5.4.3.
- Customers on XN 2.0.1 and associated versions of SNS would be rolled back to SNS 5.9.3.